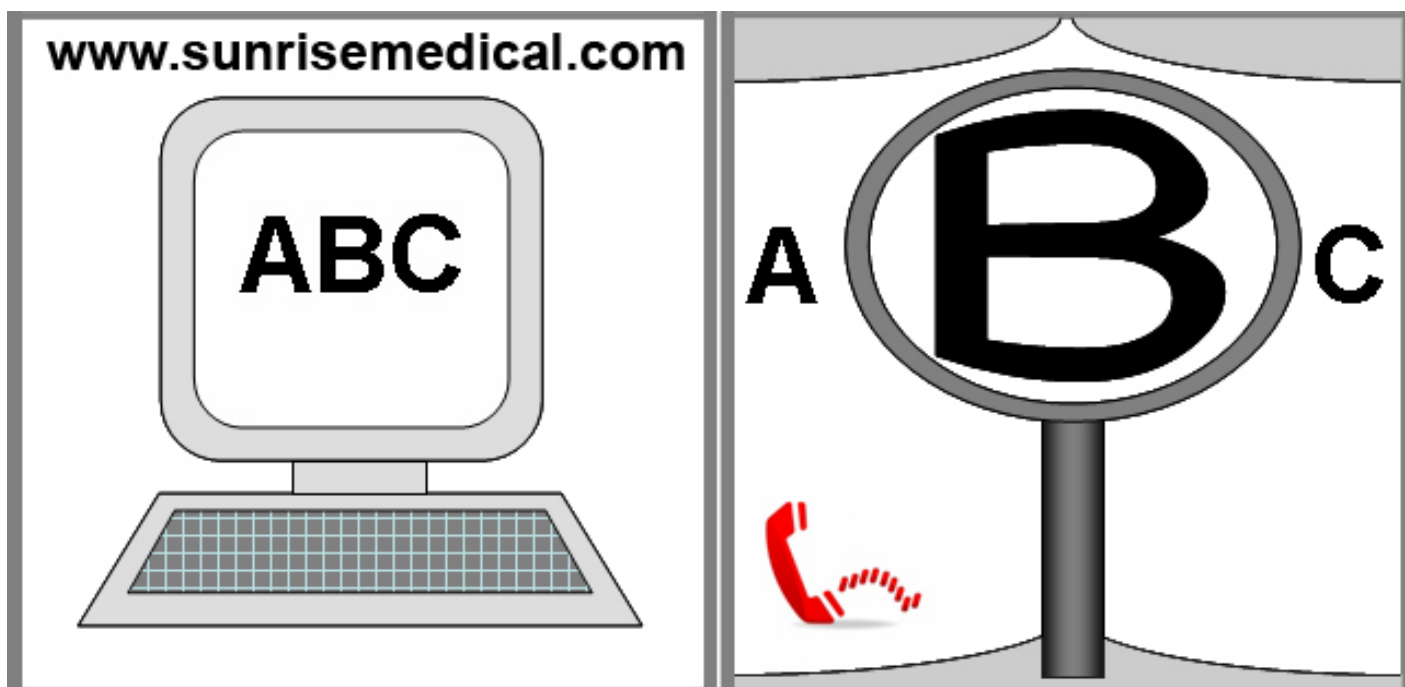
**ADD ON BIKE  
KIT HANDBIKE  
AANKOPPELFIETS**

**000690707.11**

# Attitude Power

**DIRECTIONS FOR USE  
NOTICE D'UTILISATION  
GEBRUIKERSHANDLEIDING**

**IF YOU ARE VISUALLY IMPAIRED, THIS DOCUMENT CAN BE VIEWED IN PDF  
FORMAT AT [WWW.SUNRISEMEDICAL.CO.UK](http://WWW.SUNRISEMEDICAL.CO.UK)**

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#### **Add-on Bike Components**

EN 

We at SUNRISE MEDICAL have been awarded the ISO-9001 certificate, which affirms the quality of our products at every stage, from R&D to production. This products meet the requirements in accordance with EC guidelines. Options or accessories shown are available at extra cost.

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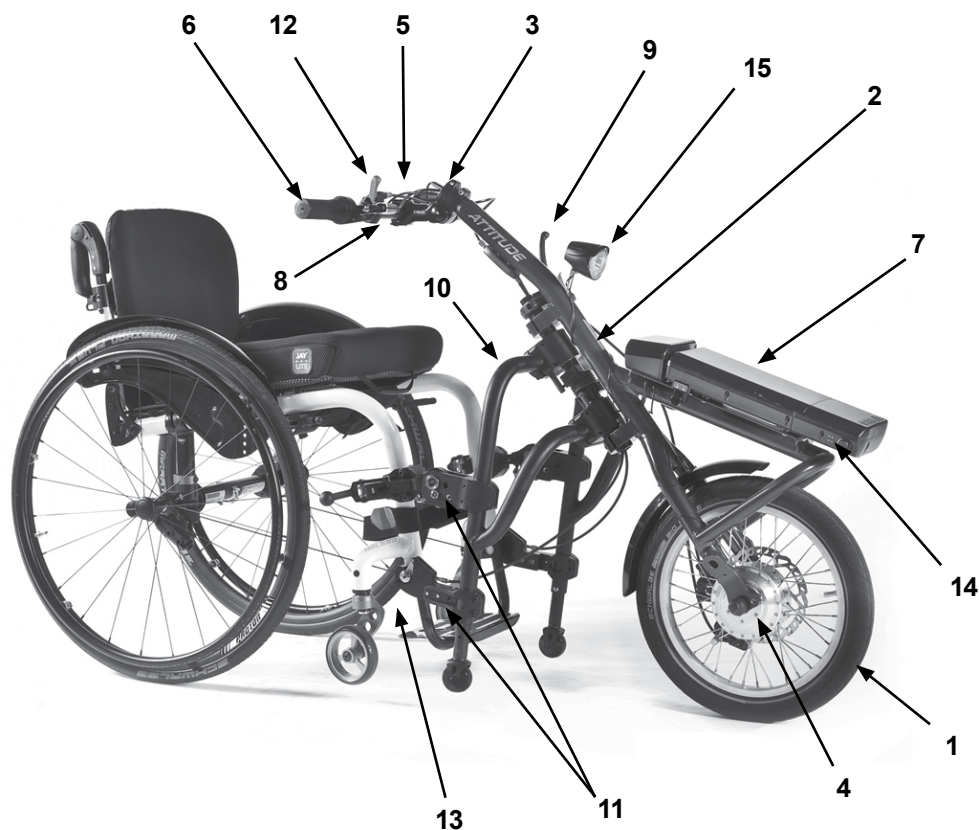
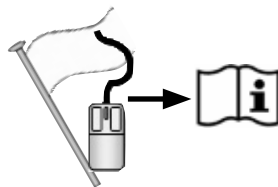
## Contents

|                                                    |           |
|----------------------------------------------------|-----------|
| <b>Foreword</b>                                    | <b>5</b>  |
| <b>Use</b>                                         | <b>5</b>  |
| <b>Area of application</b>                         | <b>5</b>  |
| <b>1.0 General safety notes and driving limits</b> | <b>6</b>  |
| Safety instructions – prior to each ride.          | 7         |
| Safety instructions – while driving                | 7         |
| Safety instructions - after your ride:             | 7         |
| <b>2.0 Warranty</b>                                | <b>8</b>  |
| GUARANTEE                                          | 8         |
| Liability                                          | 8         |
| <b>3.0 Handling</b>                                | <b>9</b>  |
| Delivery:                                          | 9         |
| Unpacking:                                         | 9         |
| Powered version:                                   | 9         |
| Start Up:                                          | 9         |
| Storage:                                           | 9         |
| Transport:                                         | 9         |
| Security:                                          | 9         |
| <b>4.0 Adjusting the Pedal Position</b>            | <b>10</b> |
| Adjustment:                                        | 10        |
| <b>5.0 Adjusting the Connecting Frame</b>          | <b>12</b> |
| Adjustment:                                        | 12        |
| <b>6.0 Docking and Dedocking</b>                   | <b>13</b> |
| Docking: Fig.6.1 - Fig.6.6                         | 13        |
| De-Docking: Fig.6.6 - Fig.6.10                     | 14        |
| <b>7.0 Functional Components</b>                   | <b>14</b> |
| Connecting frame (next page Fig.7.1):              | 14        |
| Parking brake (next page Fig.7.2 ):                | 14        |
| Main brakes (Fig.7.3):                             | 15        |
| Performance brake lever (Fig.7.4):                 | 15        |
| Throttle:                                          | 16        |
| <b>8.0 Maintenance</b>                             | <b>17</b> |
| <b>9.0 Service Period</b>                          | <b>18</b> |
| Servicing:                                         | 18        |
| <b>10.0 Tyre Maintenance</b>                       | <b>18</b> |
| Fitting and repairing tyres:                       | 18        |
| Inflation:                                         | 18        |
| <b>11.0 Maintenance Brakes</b>                     | <b>18</b> |
| Maintaining the Brakes:                            | 18        |
| <b>12.0 Cleaning/Hygiene</b>                       | <b>18</b> |
| Hygiene when being reused:                         | 18        |
| <b>13.0 Disposal / Recycling of Materials</b>      | <b>19</b> |
| Materials Used:                                    | 19        |
| <b>14.0 Nameplate</b>                              | <b>19</b> |
| <b>15.0 Torque</b>                                 | <b>20</b> |
| Torque Settings:                                   | 20        |
| <b>16.0 Technical Data</b>                         | <b>20</b> |

## Definitions of words used in this manual

| Word                                                                                              | Definition                                                                                    |
|---------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|
|  <b>DANGER!</b>  | Advice to the user of potential risk of serious injury or death if the advice is not followed |
|  <b>WARNING!</b> | Advice to the user of a risk of injury if the advice is not followed                          |
|  <b>CAUTION!</b> | Advice to user that potential damage to equipment may occur if the advice is not followed     |
| <b>NOTE:</b>                                                                                      | General advice or best practice                                                               |
|                  | Reference To Additional Documentation                                                         |

### NOTE:

- Please keep a note of your local service agent's address and telephone number in the space provided.
- In the event of a breakdown, contact them and try to give all relevant details so they can help you quickly.
- The Add-on Bikes shown and described in this user guide may not correspond in every detail exactly to your own model. However, all instructions are still entirely relevant, irrespective of detail differences.
- The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

Dealer signature and stamp

## Foreword

### Dear Customer,

We are very happy that you have decided in favour of a high-quality product from SUNRISE MEDICAL.

This owner's manual will provide numerous tips and ideas so that your new Add-on Bike can become a trustworthy and reliable partner in your life.

For Sunrise Medical, it is very important that we have a good relationship with our customers. We like to keep you up-to-date about new and current developments at our company. Keeping close to our customers means: fast service, as little red tape as possible, working closely with customers. When you need replacement parts or accessories, or if you just have a question about your Add-on Bike – we are there for you.

We want you to be satisfied with our products and service. At Sunrise Medical we are constantly working to develop our products further. For this reason, changes can occur in our palette of products with regard to form, technology and equipment. Consequently, no claims can be construed from the data or pictures contained in this user's manual.

**The management system of SUNRISE MEDICAL is certified to EN ISO 9001, ISO 13485 and ISO 14001.**

 **As the manufacturer, SUNRISEMEDICAL, declares that the add-on bikes conform to the 93/42/EEC directive.**

Please contact your local, authorised SUNRISE MEDICAL dealer if you have any questions concerning the use, maintenance, or safety of your Add-on Bike.

In case there is no authorised dealer in your area, or you have any questions, contact Sunrise Medical either in writing or by telephone.

**Sunrise Medical**  
**Thorns Road**  
**Brierley Hill**  
**West Midlands**  
**DY5 2LD**  
**England**  
**Phone: 0845 605 66 88**  
**Fax: 0845 605 66 89**  
**www.SunriseMedical.co.uk**

Do not use your Add-on Bike until this Owner's Manual has been read and understood. Please follow the instructions contained within the Owner's Manual.

## Use

The ATTITUDE is a powered drive unit, which is attached to a wheelchair, to support the mobility and integration of the wheelchair user in daily life. It is designed for personal use indoors or outdoors.

**The maximum weight limit (includes both the user and any weight of accessories fitted to the Add-on Bike) is marked on the serial number label. The serial number label is located on the frame as well as in the owner's manual.**

Warranty can only be taken on if the product is used under the specified conditions and for the intended purposes.

### Lifetime

The expected life of the add-on bike is 5 years, provided that:

- It is used in strict accordance with the intended use.
- All service and maintenance requirements are fulfilled.

### DANGER!

- Do not fit any unauthorized electronic equipment.

## Area of application

The Add-on Bike provides the user with the potential to safely cover long distances in an ergonomically and ecologically efficient manner, using their own wheelchair. It expands the activity radius significantly.

In general we recommend the use of a rear wheel extension or additional weights to improve the traction of the driving wheel especially when driving uphill. The attached wheelchair must be equipped in accordance with road traffic regulations when used on public roads, pavements and spaces.

The max user weight is 100 kg. The max load for transported objects (e.g. in bike panniers) is 5kg each.

Please also check the Owner's Manual of the bike panniers.

The add-on bike can be operated by people who:

- are physically and mentally capable of safely operating the device in all operating situations (steering, braking, pedalling...) and can meet the legal requirements for use on public roads.
- Have been advised and trained in its use by Sunrise Medical or an authorized Sunrise Medical dealer

### Operating Conditions

The ATTITUDE should be able to cope with tarmac surfaces and firm paths with different topography. Avoid driving on unpaved or loose surfaces (e.g. on loose gravel, in sand, mud, snow, ice or through deep puddles of water), as this may expose the user to unforeseen risks.

## 1.0 General safety notes and driving limits

The technology and construction of this add-on bike have been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded in parts. Nevertheless, users may put themselves at risk by improperly using their add-on bikes. For your own safety, the following rules must be strictly observed. Unprofessional or erroneous changes or adjustments increase the risk of accident. As an add-on bike user, you are also part of the daily traffic on streets and pavements, just like anyone else. We would like to remind you that you are therefore also subject to any and all traffic laws. Be careful during your first ride in this add-on bike. Get to know your add-on bike.

### **WARNING!**

- NEVER exceed the maximum load of 100 kg for driver excluding any items carried on the add-on bike.
- If the max user weight of the attached wheelchair is less than 100 kg, this lower limit will be valid for the complete system.
- Exceeding the maximum load can lead to damage to the add-on bike, loss of control, or serious injury to the user and other people.
- Exercise caution when using the add-on bike. For example, avoid negotiating obstacles such as, steps, curbs, path edges or dropping down gaps without slowing down or braking first.
- Your add-on bike has not been designed for transportation in a vehicle. DO NOT sit on the add-on bike during any transportation in a vehicle; always use a properly secured vehicle seat!!
- This add-on bike should be used exclusively to convey one person at a time. Any other use does not comply with the intended purpose.
- Use of the add-on bike is only recommended for those who have the right physical and psychological constitution to be able to cope with public traffic.
- Please note that you are subject to any and all traffic laws when riding in public traffic.
- As a beginner, you should ride particularly carefully at first.
- Familiarise yourself with the braking distances from various speeds.
- Familiarise yourself with the general traffic laws!
- In order to achieve a stable ride when travelling straight ahead, wherever possible, you should avoid hasty steering movements when travelling at speed or going round corners.
- A three-wheeled device is always less stable than a four-wheeled device.
- When you are riding, braking or manoeuvring, always hold the handlebars firmly with both hands. Failure to do so could increase the risk of an accident through steering error.
- The add-on bike is a powered drive unit. The speed must be adjusted to suit the technical capabilities of the bike, terrain and operator. The max speed is 6.0 km/h. Never exceed this max speed.
- At night and dawn/dusk, always use the lights.
- When you are travelling long distances, we recommend that you take a spare battery with you.

- Only ride on asphalt paths, or firm, smooth surfaces.
- Use caution when riding over curbs, steps, potholes, etc. as there is an increased risk of tipping over.
- Adjust your method of driving to your capabilities.
- Ride slowly and carefully, particularly when riding round corners.
- Do not touch the motor, because it can get very hot
- Your bike should be serviced by your dealer at regular intervals. (min. every year)
- As with all moving parts, there is a risk that fingers could become trapped. Please always handle carefully.
- If the add-on bike is subject to direct sunlight / low temperatures for long periods, parts of your add-on bike could become very hot (>41°C) or very cold (<0°).
- Always make sure that the quick tensioner on the drive wheel is set properly.
- The add-on bike should not be used in storm conditions, heavy rain/snow or on slippery or damaged surfaces.
- You should only use product combinations which have been approved by Sunrise Medical.

### **DANGER!**

**CHOKING HAZARD** – This mobility aid uses small parts which under certain circumstances may present a choking hazard to small children.

The wheelchairs shown and described in this user guide may not correspond in every detail exactly to your own model. However, all instructions are completely relevant, regardless of possible detail differences.

**NOTE:** The add-on bikes shown and described in this user guide may not correspond in every detail exactly to your own model. However, all instructions are completely relevant, regardless of possible detail differences. The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

**NOTE:** We would like to point out that using the 23 kph version of the Attitude Power is only permitted in road traffic in certain countries and otherwise may only be used on private land.

In the absence of approval, in accordance with the road traffic laws, the Attitude Power 23 kph version may not be driven on public roads or even cycle ways. You need to inform yourself about the country specific legislation about the use and the insurance duty of the Attitude Power 23 kph version.

When placing an order, please confirm to us with your signature that you have understood this information and that we should deliver the 23 kph version of the Attitude Power.

Example:

"If I want to use the Attitude Power version 23 kph in another country, I will first obtain information about the regulations and legislation in force there": Signature.

## Safety instructions – prior to each ride.

### CAUTION!

- Check the conditions of your wheels, (including the wheels of your wheelchair) visually, (spokes and rims, damages of tyres) as well as the pressure of the tyres. The correct pressure is indicated on the tyre and for the add-on bike is normally 3-4 bar.
- Low tyre pressure increases the risk of tipping over especially in bends, it also effects the braking action, the wear and tear and the driving performance due to increased rolling resistance.
- Always check that the tread depth of your tyres is Okay.
- Check the function of the brakes; the add-on bike may not be used if the brakes are not 100% functioning!
- Avoid damage to the Bowden cables for the gear-shift and braking units. There is a risk of kinking the cable when you transfer to/from the bike.
- Check the fork by visual inspection for damage (e.g. cracks and breakages)
- Check that all components, especially the bolts, are tight.
- Check that the handlebars are firmly fixed.
- Check the connection between the add-on bike and your wheelchair. Clamping must be on the metal tube without any parts, (e.g. calf strap, frame protectors), in between.
- Check the frame and the components of your wheelchair for any damage, (e.g. cracks)
- Check that the lights are functioning and the reflectors are sound; they must be visible all the time during use.
- Ensure that your feet cannot slip off the foot plate of your wheelchair, if necessary use a special fixation device.
- Check all cables and electrical connections.
- Remember, the max load for the saddle bags is 5 kg each.
- Check that the battery is locked correctly and fully charged.
- Switch the battery on and check the light.
- Always wear a helmet.

## Safety instructions – while driving

### WARNING!

- Make yourself familiar with the product in flat terrain before driving up or down slopes.
- Always hold the handlebars with both hands firmly as otherwise, there is a high risk of an accident through steering error.
- Always adapt your speed to your driving capability and to the traffic - / terrain conditions
- Use particular caution when approaching stairs, edges, drops or other hazard areas.
- Reduce your speed to walking pace and lean your body inwards, when driving round a bend.
- When waiting e.g. at a pedestrian crossing, a traffic light, on hills or slopes or at ramps of any type, always activate the brake.

- Due to the increased turning circle, turning can be sometimes impossible e.g. in corridors and other confined spaces.
- Only use the defined brackets to fix objects, (additional weights, bike panniers)
- When driving on pedestrian areas, observe the maximum permitted speed (walking speed)
- Always follow the road traffic regulations when driving on public roads and footpaths,
- Avoid driving on unpaved or loose surfaces (e.g. on loose gravel, in sand, mud, snow, ice or through deep puddles of water).
- Avoid abrupt steering actions
- Avoid driving in parallel to slopes
- Avoid turning on slopes
- On slopes the traction of the driving wheel is lowered and the braking effect is considerably reduced. The driving style and speed should be always adjusted so that the product can be safely stopped at all
- Times without any risk.
- The traction can be improved by using an additional weight or the rear wheel extension.
- Towing or using a trailer is not permitted.
- Driving on stairs is not permitted.
- Obstacles e.g. curbs must be run over always frontal / rectangular with very low speed to avoid tipping over and potential injuries. The max obstacle height is 50 mm, but this depends on the distance of the stand wheels and castors to the ground and therefore of the adjustments of the add-on bike – wheelchair system.
- It must be avoided that any wheelchair components or parts of your body get trapped when driving over or passing obstacles. This could result in falls and serious injuries as well as damage to the
- Add-on bike and the wheelchair.
- When driving on wet surfaces, there is an increase slip hazard because the traction of the tyres is reduced. Adapt your driving style accordingly.
- Always use the main brake to slow down your bike. In case the main brake will fail the parking brake can be sued as emergency brake once.
- Sliding of the driving wheel during braking can be avoided by moving your body weight forward.
- Avoid strong braking actions while driving through bends.

## Safety instructions - after your ride:

- Switch off the product immediately when it is not in use to avoid accidental operation / movements caused by accidentally touching the crank handle.
- Switching off helps to preserve the battery charge.

### THIS GUARANTEE DOES NOT AFFECT YOUR LEGAL RIGHTS IN ANY WAY.

Sunrise Medical\* provides a guarantee, as set out in the warranty conditions, for wheelchairs to its customers covering the following.

#### Warranty conditions:

1. Should a part or parts of the wheelchair require repair or replacement as a result of a manufacturing and/or material fault within 24 months or for frame and cross-braces within 5 years after delivery to the customer, then the affected part or parts will be repaired or replaced free of charge. The warranty will only cover manufacturing defects
2. To enforce the warranty, please contact Sunrise Medical Customer Service with the exact details of the nature of the difficulty. Should you be using the wheelchair outside the area covered by the Sunrise Medical customer service agent, repairs or replacement will be carried out by another agency as designated by the manufacturer. The wheelchair must be repaired by a Sunrise Medical designated Customer Service agent, (dealer).
3. For parts, which have been repaired or exchanged within the scope of this warranty, we provide a warranty in accordance with these warranty conditions for the remaining warranty period for the wheelchair in accordance with point 1).
4. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, (following the fitting), in accordance with these warranty conditions.
5. Claims from this warranty shall not arise, if a repair or replacement of a wheelchair or a part is required for the following reasons:
  - a. Normal wear and tear, which include batteries, armrest pads, upholstery, tyres, brakes shoes, etc.
  - b. Any overloading of the product, please check the EC label for maximum user weight.
  - c. The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the user instructions and/or the service instructions.
  - d. Accessories have been used which are not specified as original accessories.
  - e. The wheelchair or part having been damaged by neglect, accident or improper use.
  - f. Changes/modifications have been made to the wheelchair or parts, which deviate from the manufacturer's specifications.
  - g. Repairs have been carried out, before our Customer Service has been informed of the circumstances.
6. This guarantee is subject to the law of the country in which the product was purchased from Sunrise Medical\*

\* Means the Sunrise Medical facility from which the product was purchased.

#### Additional Notes For Australia Only:

- i. For goods provided by Sunrise medical Pty Ltd in Australia, our goods come with a guarantee by Sunrise Medical that cannot be excluded under Australian Consumer Law.
- ii. You are entitled to a replacement or refund for a major failure and for compensation for any foreseeable loss or damage.
- iii. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.
- iv. The benefits to you given by this warranty are in addition to your other rights and remedies under a law in relation to the goods to which the warranty relates.

#### Liability

In case that the add-on bike is:

- handled improperly
- not maintained in accordance with the maintenance schedule
- commissioned and used contrary to the instructions in this manual
- repairs or other works are carried out by non-authorised persons
- third-party parts are installed or connected without prior agreement with Sunrise Medical.

Sunrise Medical GmbH is not responsible for the safety of the add-on bike if any of the above applies.

### 3.0 Handling

#### Delivery:

Your new add-on bike is delivered fully assembled in a cardboard box. In order to avoid any damage during transportation, loose parts or parts to be fitted, are packed separately in the cardboard box.

#### Unpacking:

- Check the shipping cardboard box for external damage which may have been caused during transportation.
- Remove all packaging material.
- Take the add-on bike carefully out of the box and check that it is complete.
- Check all parts for surface damage, scratches, cracks, dents, distortions and other defects.
- The scope of delivery consist usually of the ATTITUDE add-on bike:

#### Powered version:

Connecting frame with complete docking system, display, front light, Lithium Ion battery and charger.

#### Start Up:

To start the add-on bike you need to switch on the power supply at the battery before each ride. Therefore press the on/off switch at the battery, (Fig.3.1).

**Fig.3.1**

**NOTE:** Should you find any damage, or if any parts are missing please let us know immediately.

#### ⚠ CAUTION!

- The add-on bike may only be used with the delivered docking system and the suitable docking pins for your wheelchair. The add-on bike must be handed over ready to use from Sunrise Medical or an authorized Sunrise Medical dealer.
- Before using for the first time, the battery should be fully charged.

#### Storage:

The add-on bike should always be stored in an environment with low humidity, on an easy to clean surface at ambient room temperature (+15°C to + 25°C).

#### ⚠ CAUTION!

To avoid creating a hazard the Lithium Ion battery, must be stored according the description given in the section Part 2 Ansmann Battery.

#### Transport:

Your add-on bike has not been designed for transportation in a vehicle.

#### ⚠ WARNING!

- DO NOT sit on the add-on bike during transportation in a vehicle.
- Always use a properly secured vehicle seat.
- The product and all its components must be secured during transport so that they are not damaged (e.g. by falling over) and do not become any risk to other people.

#### ⚠ CAUTION!

- When loading make sure that the cables are not caught up, become kinked or otherwise damaged.
- The add-on bike may not be used with damaged cables.

#### Security:

- To protect your Attitude against theft and also against unauthorized use, please use a suitable lock, such as those designed for bicycles.
- For extra security on the Attitude Power, please switch off and remove the battery.

## 4.0 Adjusting the Pedal Position

### Adjustment:

The handlebar position can be adjusted to the user (arm length and body position) by an authorized dealer according the available adjustment options.

### ⚠ WARNING!

- The handlebar must not touch the knee or thigh while driving, especially through bends.
- Do not allow the elbows to completely extend when the handlebars are furthest away from the body.
- If you have poor seating stability, or weak trunk muscles, a suitable restraint must be used. Please contact your therapist, doctor or authorized dealer.

The handlebar height can be adjusted by 55 mm by changing the angle of the stem, (Fig.4.1). In addition the handlebar position can be adjusted via the docking system by at least 50 mm forward and 30 mm upward, (Fig.4.2).

Examples of the possible adjustment range, and the differences due to the given wheelchair dimensions / settings, (Fig.4.3 - 4.6).

### ⚠ WARNING!

- Ensure that all cables are free from entrapment and kinks.

Fig.4.1

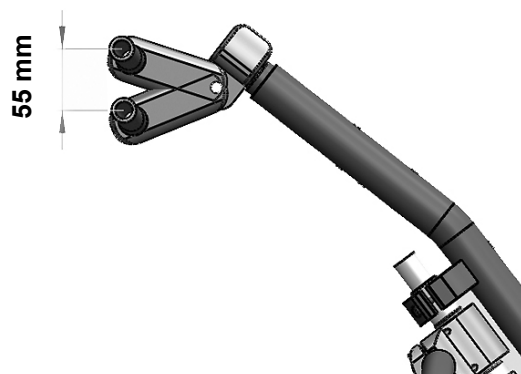
Fig.4.2

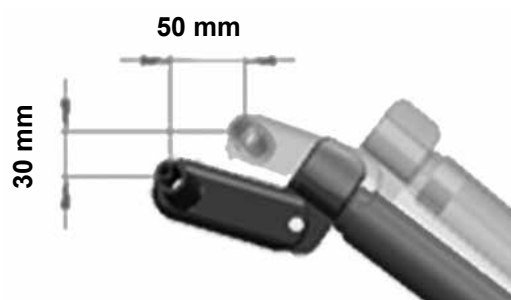
Fig.4.3


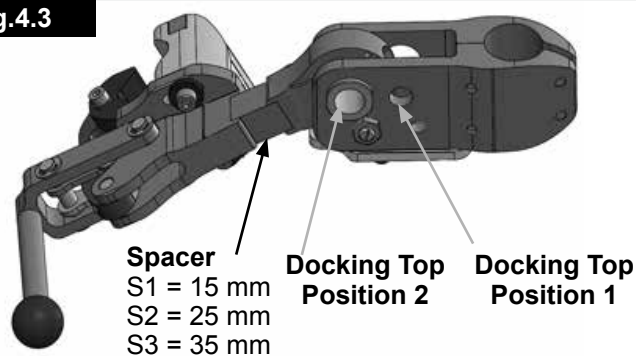
Fig.4.4

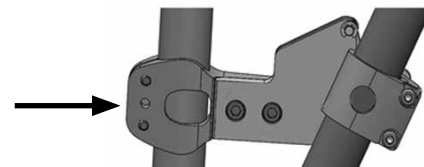Docking Bottom Position 1.

Docking Bottom Position 2.

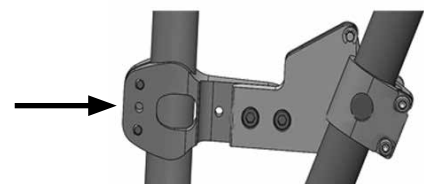
Fig.4.5



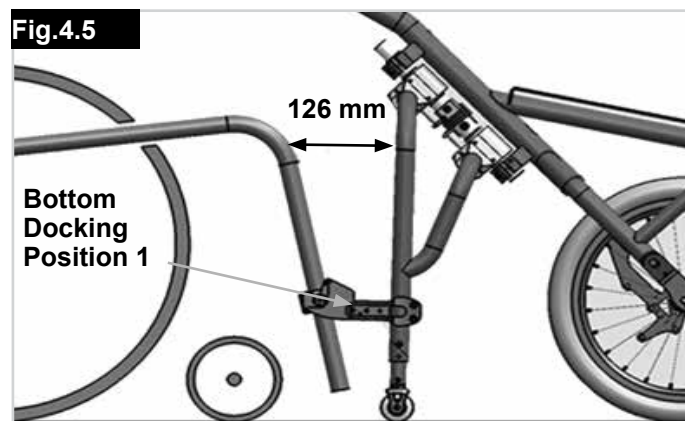
Fig.4.6

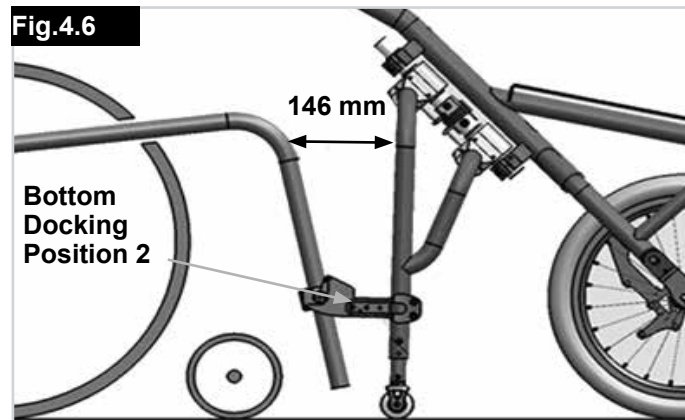The adjustment range is based on a wheelchair with affront frame angle of 92°, SD 44, SH front 500 mm, SH rear 450 mm and a COG of 80.0 mm, (Fig.4.7 and Table).

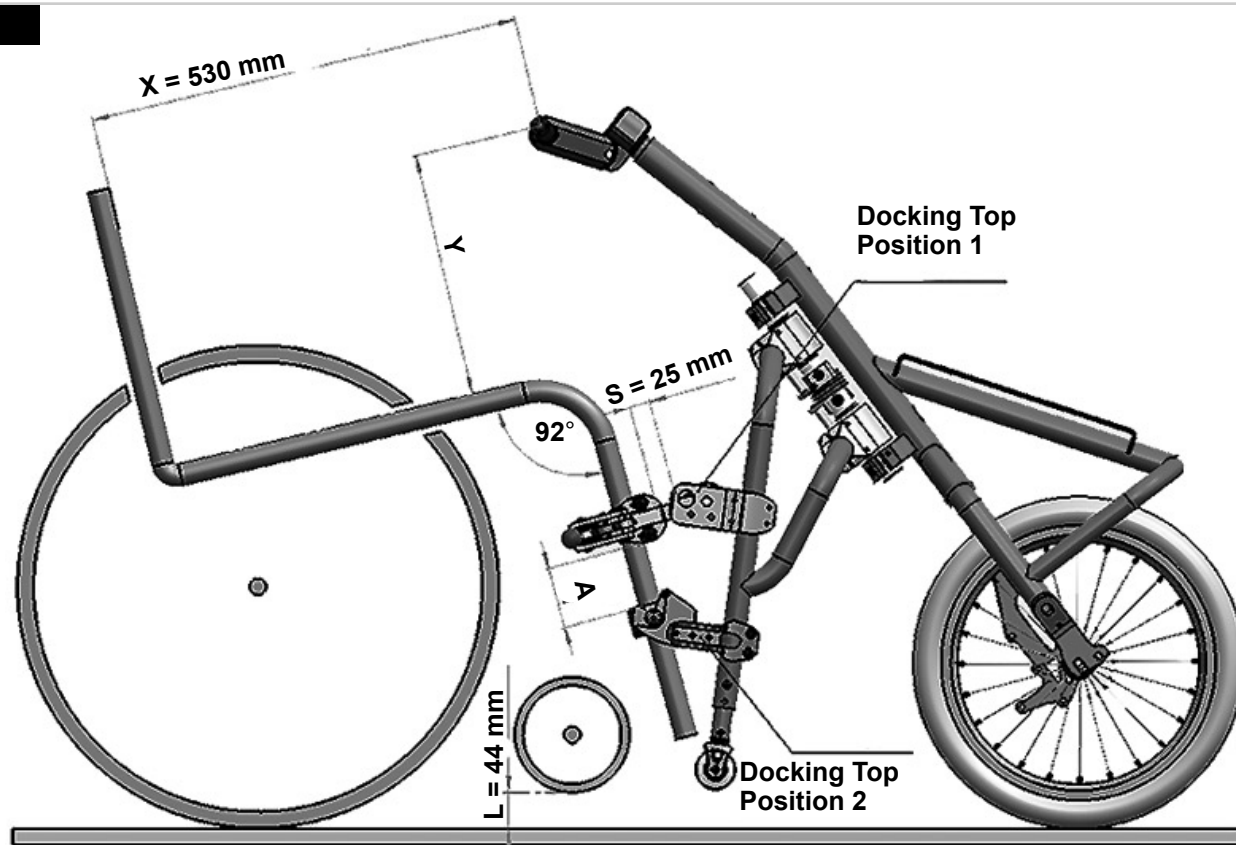Fig.4.7

| Top Dock   | Bottom Dock | X      | Y      | S     | A          | L     |
|------------|-------------|--------|--------|-------|------------|-------|
| Position 1 | Position 1  | 500 mm | 300 mm | 25 mm | 75 - 95 mm | 35 mm |
| Position 1 | Position 1  | 510 mm | 300 mm | 25 mm | 75 - 95 mm | 37 mm |
| Position 1 | Position 1  | 520 mm | 300 mm | 25 mm | 75 - 95 mm | 41 mm |
| Position 1 | Position 1  | 530 mm | 300 mm | 25 mm | 75 - 95 mm | 44 mm |
| Position 1 | Position 2  | 540 mm | 300 mm | 15 mm | 75 - 95 mm | 47 mm |
| Position 1 | Position 2  | 550 mm | 300 mm | 15 mm | 75 - 95 mm | 50 mm |
| Position 2 | Position 2  | 520 mm | 290 mm | 15 mm | 75 - 95 mm | 35 mm |
| Position 2 | Position 2  | 530 mm | 290 mm | 15 mm | 75 - 95 mm | 39 mm |
| Position 2 | Position 2  | 540 mm | 290 mm | 25 mm | 75 - 95 mm | 42 mm |
| Position 2 | Position 2  | 550 mm | 290 mm | 25 mm | 75 - 95 mm | 45 mm |
| Position 2 | Position 2  | 560 mm | 300 mm | 25 mm | 75 - 95 mm | 49 mm |

## 5.0 Adjusting the Connecting Frame

### Adjustment:

The connecting frame can be extended by approximately 20mm to fit a wheelchair with a wider front frame. Therefore the 12 allen screws on the top and bottom cover of the adjuster clamp need to be loosened. The connecting frame can then be pulled out on both sides, (Fig.5.1).

### ⚠ CAUTION!

- Ensure that the connecting frame is extended equally on both sides
- Never extend the connecting frame too far.
- Ensure that the ends of the frame tube are within the "MAX" marking printed on top of the adjuster clamp, (see Fig.5.2).

Fig.5.1


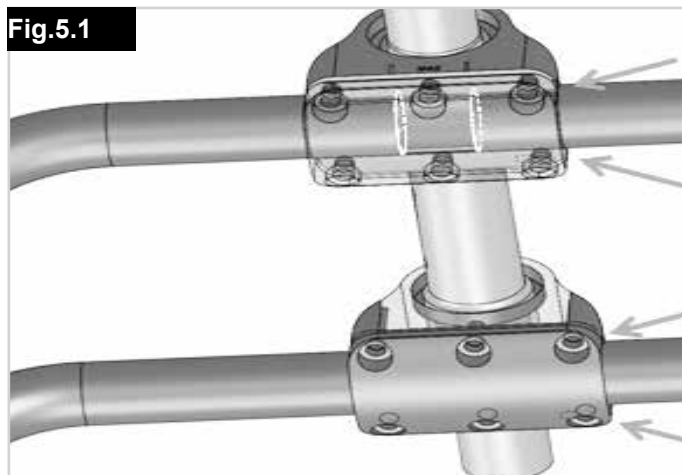
Fig.5.2
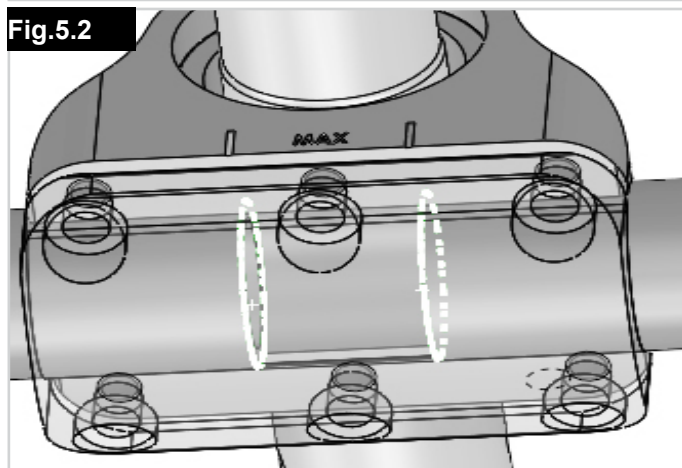
## 6.0 Docking and Dedocking

### ⚠ WARNING!

- Always make sure that the device is switched off before docking and dedocking.
- Ensure the wheelchair is fully connected with the add-on bike before use.
- As with all moving parts, there is a risk that fingers and / or clothing could become trapped. Please always handle carefully.

Refer to the docking check list below:

|                                       |                                                                                    |   |
|---------------------------------------|------------------------------------------------------------------------------------|---|
| <b>Docking Pins Fully locked In</b>   |   | ✓ |
| <b>Docking Levers Fully Locked In</b> |  | ✓ |

Fig.6.0

### ⚠ CAUTION!

- Always keep the docking system clean.
- Clean it after every trip paying particular attention to sand and mud.

### Docking: Fig.6.1 - Fig.6.6

- Drive your wheelchair as close as possible to the add-on bike.
- The 2 docking pins on your wheelchair should stay directly in front of the lower docking connector.
- The add-on bike should stand on an even surface.
- Lock the brakes of your wheelchair.
- Pull it towards you by using the handlebars, until the 2 docking pins lock in, A.
- Now push the add-on bike upwards/forwards until the stand wheels and the castors leave the ground.
- Swing the upper docking connector downwards until it stops, B.
- Lower the add-on bike until the upper docking connector is in contact with the front frame of your wheelchair.
- Pull the red locking lever backwards until it locks firmly and completely, C.
- Clamping must be on the metal tube without any parts, (e.g. calf strap, frame protectors), in between.
- Ensure the wheelchair is fully connected with the add-on bike, (see Fig.6.0).

Fig.6.1

Fig.6.2

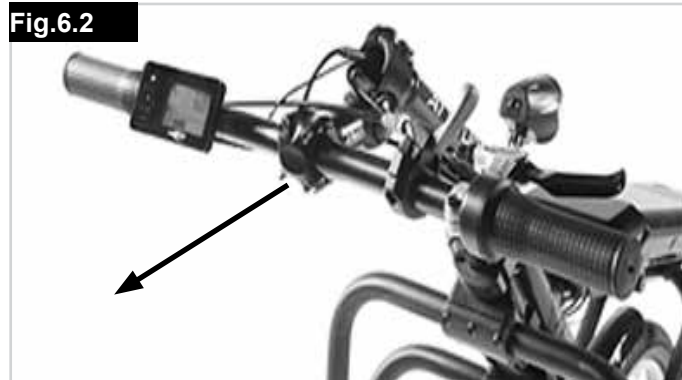
Fig.6.3

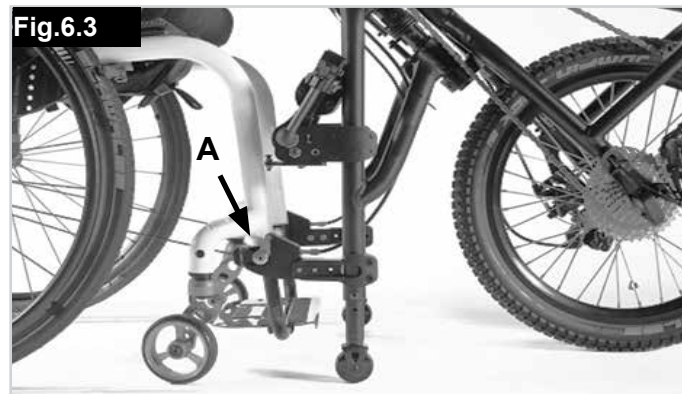
Fig.6.4


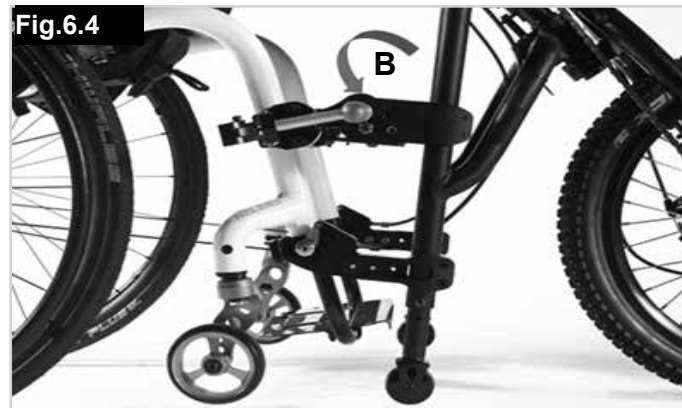
Fig.6.5
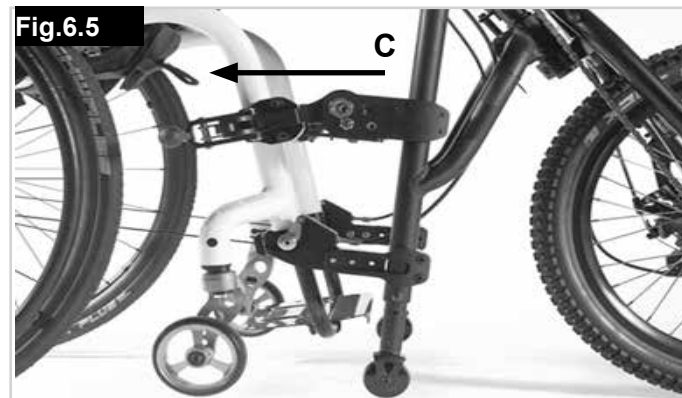
## De-Docking: Fig.6.6 - Fig.6.10


### **WARNING!**

The de-docking procedure should take place on an even surface.


**NOTE:** De-docking follows the opposite sequence as described for Docking.

- Lock the wheelchair brakes.
- Open the red locking levers, A.
- Push the add-on bike forward/upwards until the upper docking connector has no contact with the front frame of your wheelchair, B.
- Swing the upper docking connector upwards, C.
- Lower the add-on bike until the stand wheels and castors have contact to the ground.
- Pull the red release lever by the handgrip, D.
- Pull the add-on bike downwards towards yourself until the docking pins are totally disengaged.
- Hold this position and push the add-on bike forward so that it is clear of your wheelchair, E.


**Fig.6.6**

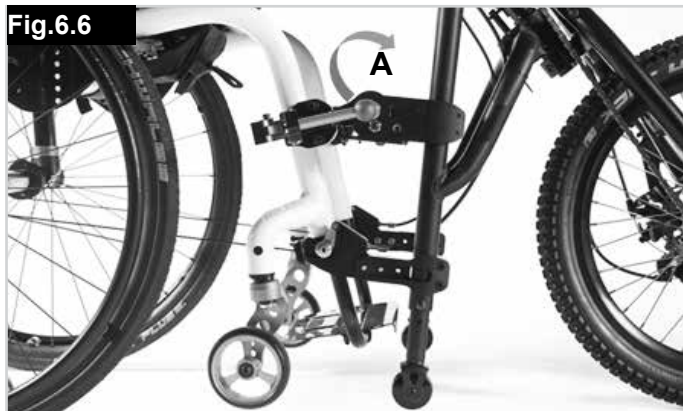
**Fig.6.7**

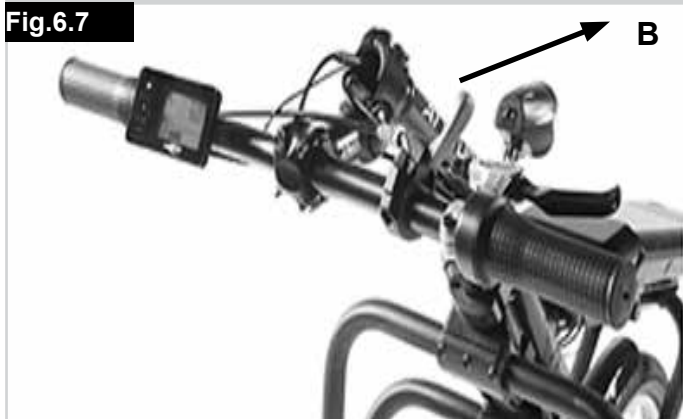**Fig.6.8**

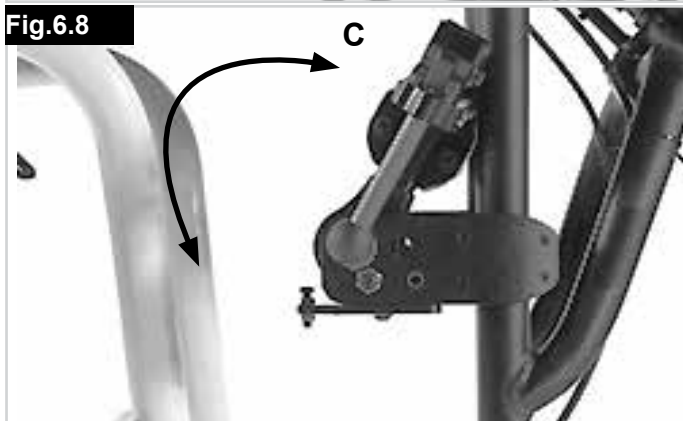**Fig.6.9**

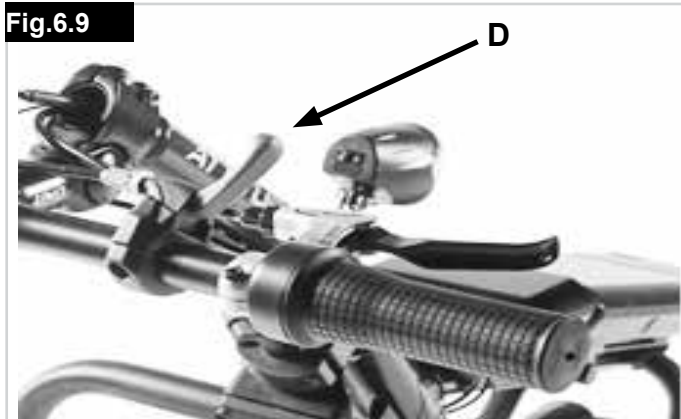**Fig.6.10**

## 7.0 Functional Components

### Connecting frame (next page Fig.7.1):

The connecting frame acts as a parking stand, and facilitates easy docking/manoeuvring when the add-on bike is disconnected from the wheelchair. The height of the stand wheels can be adjusted using the bolts, (A).

### **WARNING!**

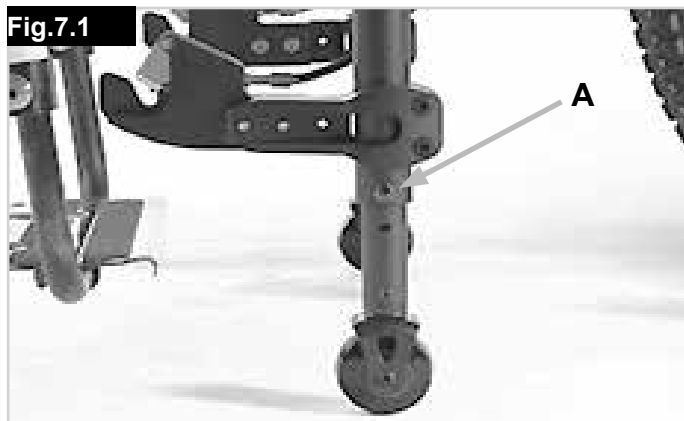
- The stand wheels must always stay min 30.0 mm above the ground, when the add-on bike is connected to the wheelchair.
- The stand wheels must be always adjusted so that the stand of the add-on bike remains stable when the add-on bike is disconnected from the wheelchair.

### Brakes:

The add-on bike has two brakes that work independently from one another.

### Parking brake (next page Fig.7.2):

The parking brake is assembled to the fork and locks in automatically when activated. Therefore the red lever must lock in. The add-on bike should not move when the parking brake is activated, ensuring parking is safe. To release the parking brake push the red lever.

**Main brakes (Fig.7.3):**

The brakes allow you to safely and comfortably brake your add-on bike.

**⚠ WARNING!**

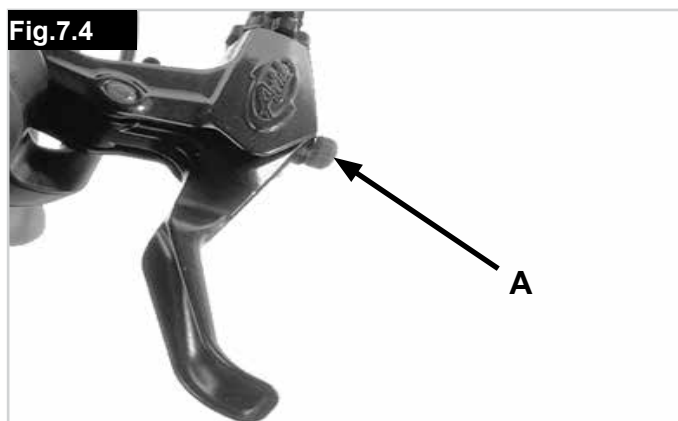
- If the main brakes should fail, then the parking brakes may be used once for emergency braking.
- The main brakes must be repaired immediately before next use.
- Hard and sudden application of the brakes may cause you to fall forward with your upper body, resulting in possible injury. Therefore always stabilise yourself by holding onto the handles firmly with both hands.
- Braking power can be severely decrease by any or all of the following:
  1. Worn tyre tread.
  2. Air pressure which is too low.
  3. Soiled and wet tyres.
  4. Worn brake pads.
- Ensure that the brake pads and brake discs do not become contaminated with oil and/or grease. If this happens the brake pads must be replaced. The brake disc must be professionally degreased with a brake cleaner.
- The disc brake is not self-adjusting and needs to be re-adjusted when the brake pads wear.
- Be aware that the condition of the ground surface can significantly affect braking efficiency.

**Performance brake lever (Fig.7.4):**

The Performance brake lever (optional) gives you the possibility to adjust the lever-feel and the way that the main brake is applied.

Turning the red adjusting screw (A) on the brake lever clockwise results in a harder lever feel and a better, linear, controllability of the brake-force.

Turning the red adjusting screw (A) on the brake lever counter-clockwise will result in a softer lever-feel and more progressive brake-force.

### **WARNING!**

- The braking effect will be greatly reduced if the surface is wet, dirty, sandy, stoney, uneven or otherwise compromised by environmental conditions. It is best to avoid such conditions, but if this is not possible adapt your driving style to avoid the risk of potentially serious accidents.
- Dirty and wet rims and brake pads also reduce the braking effect when using rim brakes. For this reason, you should always adapt your method of riding to the environmental conditions in order to avoid serious accidents.
- Always check the efficiency of the brakes before starting your journey to ensure that they are working correctly. The brakes should be checked regularly by your dealer and adjusted if required.

### **Throttle:**

The throttle lever determines the speed that can be varied between 0 km/h and max speed, (Fig.7.4).

### **WARNING!**

- Never operate the throttle when the add-on bike is not attached to the wheelchair!!
- Never operate the throttle during transfer and docking!!
- Always switch the battery off when not using the add-on bike!!

**Fig.7.4**

## 8.0 Maintenance

Before any maintenance is started, turn the battery off and remove it.

**NOTE:** All Service and Maintenance procedures carried out on the Add-on Bike require the Battery Locking Key. Without this key no maintenance is possible. **Checks to be carried out before use:**

| When                                                                                      | What                                                                                                                                                                     | Comments                                                                                                                                                                                                                                                                                                                        |
|-------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <ul style="list-style-type: none"> <li>Every 4 weeks, (depending on use).</li> </ul>      | <ul style="list-style-type: none"> <li>Grease the cables where they leave the switches and the brake levers.</li> <li>Check all screw connections are secure.</li> </ul> | <p>The following can be done by user or attendant:</p> <ul style="list-style-type: none"> <li>Before oiling, please remove old oil remnants from the components.</li> <li>Spray lightly with Teflon spray.</li> <li>Ensure that surplus oil/spray does not contaminate the surrounding area, (Clothes, brakes, etc).</li> </ul> |
| <ul style="list-style-type: none"> <li>Every 2 - 3 months, (depending on use).</li> </ul> | <ul style="list-style-type: none"> <li>Clean regularly.</li> <li>Check the unit is clean and in good condition.</li> </ul>                                               | <p>The following can be done by user or attendant:</p> <ul style="list-style-type: none"> <li>Check for cracks in the rubber components.</li> <li>Replace any cracked rubber components or take it to your nearest authorised service centre for refurbishment.</li> </ul>                                                      |

| Maintenance plan                                                                                                     | 1st inspection            | 2nd inspection            | Annual inspections              |
|----------------------------------------------------------------------------------------------------------------------|---------------------------|---------------------------|---------------------------------|
|                                                                                                                      | "250 km or after 6 month" | "1000 km or after 1 year" | "every 1000 km / or every year" |
| Check all screws / fastening elements are firmly seated                                                              | X                         | X                         | X                               |
| Functional / safety check of all lights (if fitted)                                                                  | X                         | X                         | X                               |
| Functional / safety check of the docking system (add-on bike and wheelchair)                                         | X                         | X                         | X                               |
| Clean and oil / grease all pivot points and bearings                                                                 |                           | X                         | X                               |
| Check brake-pads for wear and if necessary, re-adjust the distance from brake-pad to brake-disc.                     | X                         | X                         | X                               |
| Check Throttle Function                                                                                              | X                         | X                         | X                               |
| "Check function of main and parking brake (replace cables and brake pads if needed)"                                 | X                         | X                         | X                               |
| "Check chain, gear, gear rim for wear and tear (and replace if needed)"                                              |                           | X                         | X                               |
| "Visual inspection of the fork and connecting frame for any damage (e.g. crack, formation, deformation, etc.)"       | X                         | X                         | X                               |
| "Visual inspection of the wheelchair frame and back tubes for any damage (e.g. crack, formation, deformation, etc.)" | X                         | X                         | X                               |
| Check tyres (tread, cracks, damages) and replace if needed                                                           |                           | X                         | X                               |
| Check the spoke tension and the rim, hub for damages on the add-on bike and the wheelchair                           | X                         | X                         | X                               |
| Check all cable and electrical connectors                                                                            | X                         | X                         | X                               |
| Check tightness of steering shaft and handlebars.                                                                    | X                         | X                         | X                               |

## 9.0 Service Period

### Servicing:

If you have any questions or require any help, please contact your authorized Sunrise Medical dealer, who will be pleased to advise you with regards to servicing and repairs.

Depending on how much you have used your add-on bike, we recommend that you take it to your dealer at least every 6 months, to have your add-on bike inspected by trained personnel.

## 10.0 Tyre Maintenance

The add-on bike is equipped with a 16" drive wheel. Make sure that they are always at the recommended tyre pressure, as this will significantly affect the driving characteristics. If the tyre pressure is too low, movement and manoeuvrability is made more difficult due to the increased rolling resistance and this also increases the amount of effort required.

The optimum tyre pressure (usually 3-4 bar) is indicated on each tyre.

### Fitting and repairing tyres:

Before fitting a new inner tube:

- Inspect the base of the rim and the tyre inner wall for foreign bodies and if necessary clean thoroughly
- Check the condition of the rim tape, particularly at the valve hole. Only well fitting, strong rim tape offers long-term protection against inner tube damage through burrs and sharp spoke heads.

### Inflation:

Before inflation check all round on both sides, to make sure that the inner tube is not trapped between the tyre bead and the rim. Push the valve back gently and pull it back out, so that the inner tube achieves an optimum position around the valve.

### WARNING!

After fitting, check the tyre pressure. It is essential that you maintain regulation tyre pressure and keep the tyres in perfect condition, in order to ensure your personal safety and keep your add-on bike properly.

## 11.0 Maintenance Brakes

### Maintaining the Brakes:

Check regularly the condition of the brake pads and the brake disc. After 1500 km the brake cables should be replaced completely. The brake pads need to be replaced if the thickness measured at the thinnest point is 2,5 mm. The distance between brake-pads and disc needs to be re-adjusted as the brake-pads wear.

## 12.0 Cleaning/Hygiene

- The add-on bike should be cleaned regularly and after each bigger journey with commercially available, household cleaning agents.
- If needed only use a little water.
- Handles, display and battery may only be cleaned by wiping with a damp (not wet) cloth.
- The drive wheel can be cleaned with a soft sponge or a soft brush.
- The docking pins at the wheelchair and the docking system can be cleaned with water, a soft sponge or a soft brush (recommended after each ride).
- After cleaning please use a penetrating oil (e.g. Ballisto WD 40, Caramba...) to lubricate the moving parts of the docking.

### CAUTION!

- Never use steam or high pressure to clean the product.
- Never use any abrasive cleaning agents or aggressive, acidic cleaners.
- Make sure that water will not get into contact with the electrical connectors.

### Hygiene when being reused:

When the add-on bike is to be reused, it should be prepared carefully, and be wiped and treated with spray disinfectant on all surfaces which could come into contact with the user. If you need to do this quickly, you must use a liquid, alcohol based disinfectant suitable for medical products and devices. Please pay attention to the manufacturer's instructions of the disinfectant you are using.

### 13.0 Disposal / Recycling of Materials

**NOTE:** If the add-on bike has been made available to you as part of a charity or medical loans scheme, then it may not belong to you. If it is no longer required, then follow the instructions to return it as given by the organisation that made the add-on bike available to you.

#### Materials Used:

In the following section, there is a description of the materials used on the add-on bike, in view of the disposal or recycling of the add-on bike and its packaging. There may also be special local regulations in force with regard to disposal or recycling, these must be taken into account when disposing of your add-on bike. (This can include the cleaning or decontamination of the add-on bike prior to disposal).

- Aluminum:** Fork, wheels, connecting frame, crank, main brake lever, gearing components.
- Steel:** Docking pins, pivots knee lever docking system.
- Composite:** Handles, plugs, parking brake lever and tyres.
- Packaging:** Made of soft polyethylene, cardboard
- Battery:** Lithium Ion battery (hazardous product) Please see battery section for more detailed information.

**NOTE:** Disposal or recycling should be done through a licensed agent or authorised place of disposal. Alternatively your add-on bike may be returned to your dealer for disposal.

### 14.0 Nameplate

The nameplate is located on the frame as well as on a label in the owner's manual. The nameplate indicates the exact model designation and other technical specifications. Please provide the following pieces of information whenever you have to order replacement parts or to file a claim:

[SAMPLE]

|                                                                          |                       |
|--------------------------------------------------------------------------|-----------------------|
| SUNRISE MEDICAL GmbH & Co. KG<br>Kahlbachring 2-4<br>D-69254 Malsch - HD |                       |
| ADD ON BIKE                                                              |                       |
| Attitude                                                                 | XXXXXXXX SN ATDXXXXXX |
|                                                                          | Max XXX kg            |

|                  |                                                                                                                |
|------------------|----------------------------------------------------------------------------------------------------------------|
| ATD<br>XXXXX-XXX | Product Name/SKU Number.                                                                                       |
|                  | Maximum safe slope, depends on wheelchair/ add-on bike setting, posture and physical capabilities of the user. |
| Max XXX kg       | Load Maximum.                                                                                                  |
|                  | CE Mark                                                                                                        |
|                  | User's Guide.                                                                                                  |
|                  | Not Crash tested                                                                                               |
| XX.XX.XXXX       | Date of manufacture.                                                                                           |
| SN ATD XXXX      | Serial No                                                                                                      |

As we are committed to continually improving the construction of our wheelchairs and add-on bikes, product specifications may vary slightly from the examples illustrated. All weight/dimensions and performance data is approximate and is provided solely for guidance.

All products from Sunrise Medical comply with the 93/42/EEC directive for medical products.

All add-on bikes must be used in accordance with the manufacturer's guidelines.

Sunrise Medical GmbH & Co. KG  
 Kahlbachring 2-4  
 69254 Malsch/Heidelberg  
 Deutschland  
 Tel.: +49 (0) 7253/980-0  
 Fax: +49 (0) 7253/980-222  
 kundenservice@sunrisemedical.de  
 www.SunriseMedical.de

## 15.0 Torque

### Torque Settings:

Torque in general is 7 Nm, unless otherwise specified.

The screws used are fitted with threadlock at the factory and can be adjusted up to 5 times after fitting. After that they should be replaced or alternatively re-secured with threadlock to medium tightness..

### WARNING!

Secured nuts may only be used once.

Fig.15.1

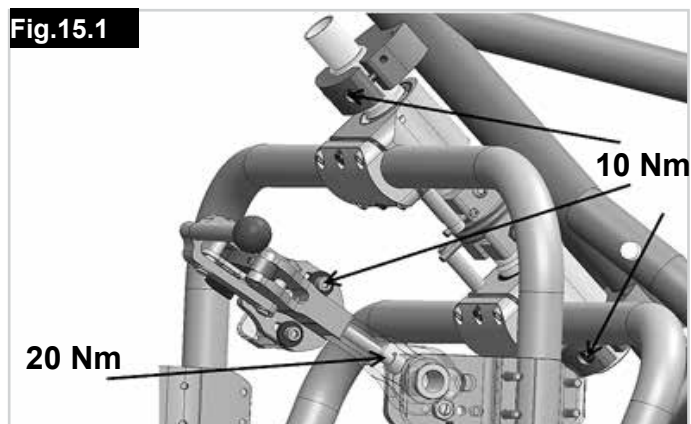
Fig.15.2

## 16.0 Technical Data

| General                            |                      |
|------------------------------------|----------------------|
| max user weight                    | 100 kg               |
| Seat width range wheelchair        | 280-460 mm           |
| Weight (cpl)                       | 13,5kg               |
| Wheel size                         | 16" (406 mm)         |
| Min Turning radius                 | 2700 mm*             |
| Min slope                          | 5° (9%)*             |
| Max. inclination (static)          | 10°*                 |
| Mix. obstacle height               | 35 mm*               |
| Max. obstacle height               | 50 mm*               |
| Motor (see also motor section 9)   |                      |
| Motor (peak)                       | 250 W                |
| Torque                             | Up to 20 Nm          |
| Range*                             | Up to 30/50 km       |
| Voltage                            | 36 Volt              |
| Speed                              | 6 km/h / 23 km/h     |
| Efficiency                         | 65%                  |
| Battery (see also battery section) |                      |
| Battery type                       | Lithium ion          |
| Battery capacity                   | 9 Ah / 13.4 Ah       |
| Protection rating                  | IP 54                |
| Rated voltage                      | 36 V                 |
| End-of-charge voltage              | 42 V                 |
| Total energy                       | 482 Wh               |
| Max discharge current              | 12 A                 |
| Ambient operating temperature      | -20 °C to 60 °C      |
| Ambient chargingg temperature      | 0 °C to 45 °C        |
| Ambient Storage temperature        | +10°C to +30°C       |
| Battery weight                     | 2.5 kg               |
| Display (LCD)                      |                      |
| Display                            | Monochrome           |
| Display Size (diagonal)            | 2,4 " (61 mm)        |
| Protection Rating                  | IP 65                |
| Operating temperature              | -20°C to + 60°C      |
| Charger                            |                      |
| Protection Rating                  | IP 65                |
| Input voltage                      | 100-240 VAC 50/60 hz |
| Operating Temperature              | 0°C - 40°C           |
| Storage Temperature                | -40°C - 80°C         |

\* Depending on the wheelchair – add-on bike combination values can vary e.g. rear wheel extension, surface conditions, user weight, driving capability, add-on weight.

# **Ansmann System**

## **Part 1 Ansmann Display**

## Contents

|                                                                                      |           |
|--------------------------------------------------------------------------------------|-----------|
| <b>Part 1 Ansmann Display</b>                                                        | <b>23</b> |
| <b>1.0 Safety Advice/Instructions</b>                                                | <b>23</b> |
| Introduction:                                                                        | 23        |
| Intended use of the Ansmann components:                                              | 23        |
| Permissible operating conditions / operating sites:                                  | 23        |
| <b>2.0 Standard Scope of Delivery</b>                                                | <b>24</b> |
| Ansmann Components:                                                                  | 24        |
| Technical Data:                                                                      | 24        |
| <b>3.0 Functions of the Ansmann system</b>                                           | <b>25</b> |
| Switching on and off:                                                                | 25        |
| Overview of functions                                                                | 25        |
| Ansmann LCD display view                                                             | 25        |
| Installing the display                                                               | 25        |
| Initial Start-up Display:                                                            | 26        |
| Back-light brightness:                                                               | 26        |
| Setting Back-light Brightness                                                        | 26        |
| Using The Display:                                                                   | 26        |
| Set Wheel Circumference:                                                             | 27        |
| Speedometer, average speed, maximum speed, total distance, trip distance, trip time: | 28        |
| Resetting the Trip Distance and Trip Time:                                           | 28        |
| <b>4.0 Drive Wheel and Motor</b>                                                     | <b>29</b> |
| Attaching the drive wheel:                                                           | 29        |
| <b>5.0 Cleaning</b>                                                                  | <b>29</b> |
| Cleaning the motor:                                                                  | 29        |
| Motor:                                                                               | 29        |
| Display:                                                                             | 29        |
| <b>6.0 Safety Precautions</b>                                                        | <b>29</b> |
| Sunlight/direct heat:                                                                | 29        |

## Definitions of words used in this manual

| Word                                                                                              | Definition                                                                                    |
|---------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|
|  <b>DANGER!</b>  | Advice to the user of potential risk of serious injury or death if the advice is not followed |
|  <b>WARNING!</b> | Advice to the user of a risk of injury if the advice is not followed                          |
|  <b>CAUTION!</b> | Advice to user that potential damage to equipment may occur if the advice is not followed     |
| <b>NOTE:</b>                                                                                      | General advice or best practice                                                               |
|                  | Reference To Additional Documentation                                                         |

Along with this operating manual, your add-on bike comes with additional documents. Please observe the specifications and instructions in these documents.

#### Introduction:

#### **WARNING!**

It is not a legal requirement at present to wear a helmet when using a add-on bike, nevertheless for your own safety, wearing a helmet is recommended!

#### Intended use of the Ansmann components:

- Your add-on bike, equipped with Ansmann components by the specialist dealer on delivery, is designed for ordinary personal transport in public road traffic use.
- Adjustments and repairs to the add-on bike and its individual components only apply as intended use insofar as they are explained and permitted in this operating manual, in the operating manual of the add-on bike manufacturer, the instructions of the component manufacturers or other documents included when purchasing your add-on bike.
- The manufacturer accepts no liability for damage caused by negligence as a result of misuse, improper maintenance or repairs or improper use.
- It is the responsibility of the rider to check the add-on bike as required, to have any work carried out on it and to use it responsibly.
- This operating manual only describes the use of the Ansmann components fitted to your add-on bike and corresponds to the state of the art at the time of print.
- The manufacturer reserves the right to make changes resulting from further development of the mechanics, software or legal requirements.

The manufacturer regards the following cases as examples of misuse of the Ansmann components fitted to your add-on bike:

- Use of the drive system that contravenes the instructions and recommendations in this operating manual.
- Exceeding the technical limits laid down in this operating manual.
- Technical modifications to the Ansmann components.
- Modifications to the software of the Ansmann components.
- Unauthorised attachments or use of the Ansmann components on bicycles or a different add-on bike to the one supplied to you.

The manufacturer shall not be liable for any damage caused by misuse of the components.

#### **WARNING!**

Before using the device, carefully read all safety and hazard information contained in the individual chapters of this operating manual and all other enclosed documents.

#### Permissible operating conditions / operating sites:

Ansmann components can be used at temperatures between -20 °C and +60 °C.

#### **WARNING!**

- Observe the information on permissible operating conditions in the ad-on manufacturer's operating instructions.
- Any limits regarding the permissible operating conditions (e.g. maximum climbing capability, maximum permissible height of obstacles, max. load, etc.) must be observed when using the add-on bike!
- Observe the safety and hazard information provided in the individual chapters of the operating manual.

## 2.0 Standard Scope of Delivery

### Ansmann Components:

- Drive motor.
- Display
- This operating manual.

### Technical Data:

#### Drive

|                                        |                                                                                            |
|----------------------------------------|--------------------------------------------------------------------------------------------|
| Range*:                                | Up to 30/50 km                                                                             |
| Speed                                  | 6 km/h (15 km/h and 23 km/h)                                                               |
| Power rating (peak)                    | 250 Watt (500 Watt)                                                                        |
| Operating voltage                      | 36 Volt                                                                                    |
| Level of efficiency                    | 65 % (incl. electronics)                                                                   |
| Control system performance electronics | Integrated into the controll box                                                           |
| Brake disc                             | From 160 mm diameter                                                                       |
| Torque receiver                        | Variable torque supports can be adjusted to the dropout                                    |
| Weight                                 | 2,5 kg (just the drive including plug and cable, without brake disc, free wheel, cassette) |

#### Display

|                                          |                               |
|------------------------------------------|-------------------------------|
| Display                                  | control monochrome            |
| Display diagonal, resolution             | 2.4 inch, (61.0 mm).          |
| Dimensions display with dock (W X L X H) | 89 mm x 50 mm x 50 mm         |
| Electrical contacting                    | Corrosion-protected contacts. |
| Illumination                             | LED backlight.                |
| Display screen                           | Plexi-glass screen            |

#### Entire system

|                       |                    |
|-----------------------|--------------------|
| Operating temperature | - 20 °C to + 60 °C |
| Protection rating     | IP65               |

**NOTE:** (\*) The range depends on the battery used and on the terrain and the prevailing travelling conditions. The specified range can be achieved at optimum travelling conditions (for instance level terrain, recently charged batteries, ambient temperature of 20 °C, smooth journey etc.) and average speed. We reserve the right to modify the design and technology of our products to incorporate the latest developments. This operating manual is available for download on our website [www.SunriseMedical.co.uk](http://www.SunriseMedical.co.uk). If you require a version that is written in a larger font, contact our Customer Service Center.

### 3.0 Functions of the Ansmann system

#### Switching on and off:

##### Switching on

To switch on the system operate the main switch at the battery. Immediately the screen appears.

##### Switching off

To switch off your add-on bike you need to press the main button on the battery.

#### ⚠ WARNING!

Always switch your add-on bike off when it is not used.

#### Overview of functions

The LCD display offers the following main functions:

- Battery voltage display
- Motor level selected
- Cycle computer : Current speed, max. speed, trip distance, daily riding time, total kilometres
- Background light (on/off)

#### Ansmann LCD display view

The 3 buttons shown below correspond to the following functions, (Fig.3.1):

**M MODE**

**+ PLUS (UP)**

**- MINUS (DOWN)**

#### Installing the display

Attach the display on the left hand side of your add-on bike's handlebar and adjust the viewing angle to suit your height. Tighten the Allen screw of the clamp so that the display does not move on the handlebar, (Fig.3.2).

#### ⚠ CAUTION!

Make sure that the distance to the speed sensor is not more than 5mm from the magnet, (Fig.3.3).

Fig.3.1

Fig.3.2

Fig.3.3

**Initial Start-up Display:**

At start-up, the screen displays all functions to confirm that the associated LCD Icon is working, (Fig.3.4).

**Back-light brightness:****NOTE:**

bL - n = Backlight Brightness then a number.

**Setting Back-light Brightness**

On the display, hold down the  and  buttons simultaneously for Three seconds to enter the Setup Menu. The display shows as Fig.3.7.

Here you can select the level of backlight in 3 steps. Using the  and  buttons you can select the brightness of your backlight. Press the M button to confirm your selection. You will automatically continue to the next menu after confirming, (Fig.3.7).

**Using The Display:**

There are 6 levels of support, that are set by using the  and  buttons. Support level "0" means that the drive system is operating, but the motor delivers no support.

To activate motor assist, press the  button and select the desired speed, (Fig.3.8).

- Level 1 ≈ 0 kp/h
- Level 2 ≈ 6 kp/h
- Level 3 ≈ 6 kp/h
- Level 4 ≈ 6 kp/h
- Level 5 ≈ 15 kp/h
- Level 6 ≈ 23 kp/h

**Fig.3.4****Fig.3.7****Fig.3.8**

## Set Wheel Circumference:

In the setting menu, "wheel circumference setting", you can adjust for the size of the wheel, so that the speedometer accurately indicates the bike speed and the odometer counts correctly, (Fig.3.9).

The wheel or tyre size is printed on the tyre.

In the table below, you can determine the correct 4-digit value and enter it into the menu.

- Press the M button to get to the next number position.
- Select each digit by pressing the  $\oplus$  and  $\ominus$  buttons.
- When you have correctly set your 4-digit wheel size, you arrive at the end of the menu.
- Hold down the M button for 3 seconds to exit the setup menu.

Fig.3.9

| E.T.R.T.O.* | Tyre Size      | Display - No. |  | E.T.R.T.O.* | Tyre Size      | Display - No. |
|-------------|----------------|---------------|--|-------------|----------------|---------------|
| 47-305      | 16x1.75x2      | 1272          |  | 32-630      | 27x1 1/4       | 2199          |
| 47-406      | 20x1.75x2      | 1590          |  | 28-630      | 27x1 1/4 Fifty | 2174          |
| 37-540      | 24x1 3/8A      | 1948          |  | 40-622      | 28x1.5         | 2224          |
| 47-507      | 24x1.75x2      | 1907          |  | 47-622      | 28x1.75        | 2268          |
| 23-571      | 26x1           | 1973          |  | 40-635      | 28x1 1/2       | 2265          |
| 40-559      | 26x1.5         | 2026          |  | 37-622      | 28x1 3/8x 15/8 | 2205          |
| 44-559      | 26x1.6         | 2051          |  | 18-622      | 700x18C        | 2102          |
| 47-559      | 26x1.75x2      | 2070          |  | 20-622      | 700X20C        | 2114          |
| 50-559      | 26x1.9         | 2089          |  | 23-622      | 700X23C        | 2133          |
| 54-559      | 26x2.00        | 2114          |  | 25-622      | 700X25C        | 2146          |
| 57-559      | 26x2.125       | 2133          |  | 28-622      | 700X28C        | 2149          |
| 37-590      | 26x1 3/8       | 2105          |  | 32-622      | 700X32C        | 2174          |
| 37-584      | 26x1 3/8x1 1/2 | 2086          |  | 37-622      | 700X35C        | 2205          |
| 20-571      | 26x3/4         | 1954          |  | 40-622      | 700X40C        | 2224          |

\* European Tyre and Rim Technical Organisation

**Speedometer, average speed, maximum speed, total distance, trip distance, trip time:**

Before, during and after a journey, various values and information are displayed and regularly saved.

By repeatedly pressing the **M** button, you can set the different views.

**Fig.3.10**

- 1. Speedometer
- 2. Average speed
- 3. Maximum speed
- 4. Trip time
- 5. Trip distance
- 6. Total kilometres


**Backlight**

To activate or deactivate the backlight of the LCD display, press and hold the **+** button for 2 seconds. You can adjust the brightness in 3 steps as described in point 3.6.

**Resetting the Trip Distace and Trip Time:**

Press the **M** button untill the setting "TRIP" is displayed. Press and hold the **M** button again for 3 seconds. This will reset both the Trip Odometer, (TRIP) and the daily travel time, (TRIP TIME), to zero.

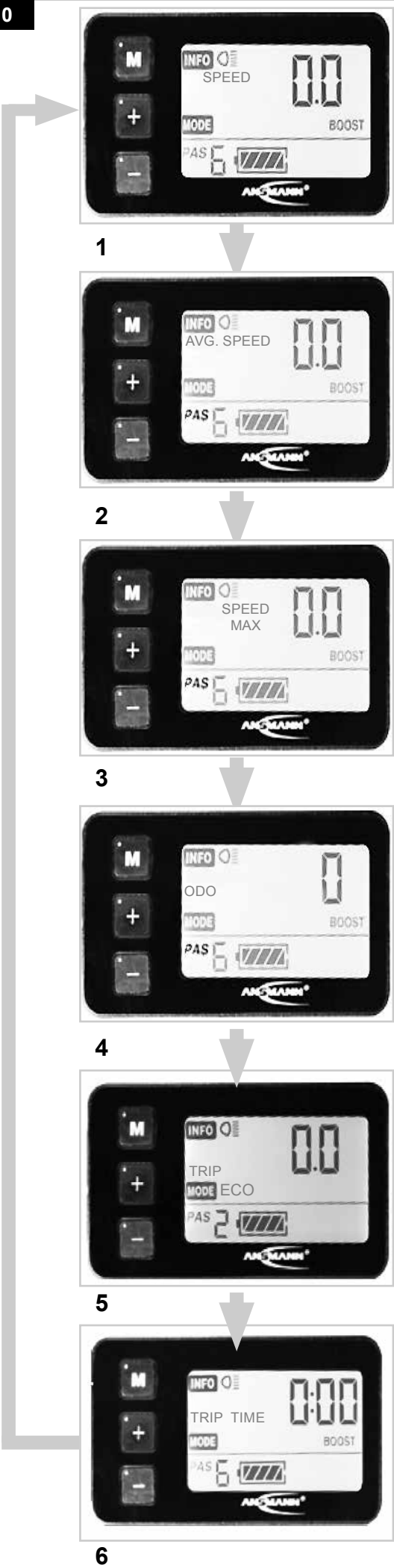**Fig.3.10**

## 4.0 Drive Wheel and Motor

### Attaching the drive wheel:

Make sure that all the components attached to the wheel have been installed in accordance with the instructions and specifications of the respective manufacturer. This relates in particular to the brakes.

Do not forget to reinstall the torque support in the same position as it was taken from.

Push the wheel into the slot of the frame and tighten it using the hub axle nuts [18] in the following sequence:

1. tighten on the side without the brakes.
2. then on the side with the brakes.

**NOTE:** the torque for the axle nuts needs to be 35 Nm

### **WARNING!**

- If the wheel is correctly attached to the frame, the motor with the end of the cable that leads to the battery can be connected. Then fasten all cables and leads with cable ties to the bicycle frame and perform a final function test.
- Make sure the cables are laid correctly as failure to do so could mean the cable getting caught in the brake disc, the drive or the spokes and by locking the wheel cause a fall.
- It is essential to observe the instructions and specifications of the manufacturer of the various components attached to the wheel in all your installation work. This particularly applies to the brakes and quick-release mechanism.
- Never install the motor without the torque support. This could cause a total loss (twisting off the cable). This voids all warranty or guarantee claims.
- Carry 5 cable ties with you along with your repair tool, to securely fasten any loose cables during a trip.
- Installing or removing the drive wheel is best carried out when the add-on bike is upside down
- (standing on its handlebar and connecting frame). Make sure the display attached to the handlebars does not get damaged!).

## 5.0 Cleaning

### Cleaning the motor:

When cleaning the device, never use cleaning benzine, thinner, acetone or similar agents. Never use abrasive detergents or aggressive cleaning agents. Instead, use only conventional household cleaning agents and disinfectants (isopropyl alcohol).

### Motor:

Your add-on bike motor should be regularly cleaned of dirt, ideally with a dry brush or a damp cloth. You also can use running water e.g. from a garden hose. Never clean the motor with a high-pressure water jet or steam. However, travelling in the rain and on wet lanes is entirely possible.

### **CAUTION!**

- Water ingress can destroy the motor. Therefore always ensure that liquid or moisture does not get into the motor.
- Do not clean the motor when it is warm, for instance, immediately after a trip. Wait until it has cooled down. Otherwise this could cause damage.
- If the motor is dismantled, for example for cleaning purposes, it must never be held or transported over the cables which would run the risk of the cable breaking.
- If the motor has been removed from the add-on bike the plug from the motor and the socket of the cable to the battery pack need to be checked for any contamination or cleaned before connecting.

### Display:

The housing may only be cleaned using a damp (not wet) cloth.

## 6.0 Safety Precautions

### Sunlight/direct heat:

When not in use, do not expose your add-on bike to strong sunlight or intense heat for long periods of time. This could cause the motor to heat up and in an extreme case, prevent it from operating at full power. Even plastic parts age quicker under intense sunlight.

# **Ansmann System**

## **Part 2 Ansmann Battery**

|                                                                              |           |
|------------------------------------------------------------------------------|-----------|
| <b>Part 2 Ansmann Battery</b>                                                | <b>32</b> |
| <b>1.0 Introduction</b>                                                      | <b>32</b> |
| Introduction:                                                                | 32        |
| Important instructions :                                                     | 32        |
| Proper use of the Ansmann battery:                                           | 32        |
| Signs and symbols:                                                           | 32        |
| Permissible operating conditions / operating sites:                          | 32        |
| Standard scope of delivery (Ansmann components):                             | 32        |
| <b>2.0 Technical Data</b>                                                    | <b>33</b> |
| The key elements at a glance:                                                | 33        |
| <b>3.0 Safety Advice/Precautions</b>                                         | <b>34</b> |
| Safety instructions and precautions on the use of the battery:               | 34        |
| Safety instructions and precautions on how to store the battery:             | 34        |
| Safety instructions and precautions on the charging process:                 | 34        |
| Safety information and precautions on how to transport and ship the battery: | 35        |
| Safety information and precautions on the charger:                           | 35        |
| <b>4.0 Battery</b>                                                           | <b>36</b> |
| Start Up:                                                                    | 36        |
| Information on the operating modes:                                          | 36        |
| Inserting the battery:                                                       | 36        |
| Switching on the battery:                                                    | 36        |
| Battery capacity indicator:                                                  | 36        |
| Detaching the battery:                                                       | 36        |
| Switching off the battery:                                                   | 36        |
| Removing the battery:                                                        | 36        |
| Charging the battery:                                                        | 37        |
| Connecting the battery charger:                                              | 37        |
| Charging process:                                                            | 37        |
| LED displays during the charging process:                                    | 37        |
| Fuse Replacement At The Battery:                                             | 38        |
| Cleaning the battery:                                                        | 38        |
| Battery Storage:                                                             | 38        |
| <b>5.0 Disposal</b>                                                          | <b>39</b> |
| Disposal:                                                                    | 39        |
| <b>6.0 Battery Charger</b>                                                   | <b>40</b> |
| <b>7.0 Liability</b>                                                         | <b>40</b> |
| Liability:                                                                   | 40        |

## Part 2 Ansmann Battery

### 1.0 Introduction

#### Introduction:

#### Important instructions :

Along with this operating manual, your add-on bike comes with additional documents. Please observe the specifications and instructions in these documents.

#### Proper use of the Ansmann battery:

The Ansmann battery is used exclusively to power the Ansmann drive system. No other components may be connected to it. Any other use requires the written authorisation of the manufacturer.

This operating manual only describes the use of the Ansmann battery and corresponds to the state of the art at the time of print.

The manufacturer reserves the right to make changes resulting from further development of the mechanics, software or legal requirements. The manufacturer regards uses including the following as misuse of the battery:

Use of the battery that contravenes the instructions and recommendations in this operating manual.

Exceeding the technical limits laid down in this operating manual.

Technical changes to the battery.

Changes to the software of the battery.

Unauthorised attachments or use of the battery.

The manufacturer shall not be liable for any damage caused by improper use of the battery.

Before using the device, carefully read all safety and hazard information contained in the individual chapters of this operating manual and all other enclosed documents.

#### Signs and symbols:

Important information for your safety are identified in this operating manual as follows:

| Word                                                                                              | Definition                                                                                    |
|---------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------|
|  <b>DANGER!</b>  | Advice to the user of potential risk of serious injury or death if the advice is not followed |
|  <b>WARNING!</b> | Advice to the user of a risk of injury if the advice is not followed                          |
|  <b>CAUTION!</b> | Advice to user that potential damage to equipment may occur if the advice is not followed     |
| <b>NOTE:</b>                                                                                      | General advice or best practice                                                               |
|                  | Reference To Additional Documentation                                                         |
|                 | Indicates magnetic forces                                                                     |

Observe these indications at all times to avoid injury to people and damage to the product.

#### Permissible operating conditions / operating sites:

#### **CAUTION!**

The battery must only be used at ambient temperatures between -20 °C and +60 °C.

Observe the information on permissible operating conditions in the add-on bike manufacturer's operating instructions. The manufacturer's restrictions on the permissible operating conditions must be observed.

Observe the safety and hazard information provided in the individual chapters of the operating manual.

#### Standard scope of delivery (Ansmann components):

- Battery including 1 key pair.
- Battery slide rail to fit the battery (already installed on the add-on bike).
- This operating manual

## 2.0 Technical Data

|                                |                 |                 |
|--------------------------------|-----------------|-----------------|
| Battery type:                  | Lithium ion     | Lithium ion     |
| Rated capacity:                | 9 Ah            | 13,4 Ah         |
| Rated voltage:                 | 36 V            | 36 V            |
| End-of-charge voltage          | 42 V            | 42 V            |
| Total energy:                  | 482Wh           | 482 Wh          |
| Continuous discharge current:  | 13 A            | 13 A            |
| Ambient charging temperature:  | 0 °C to 40 °C   | 0 °C to 40 °C   |
| Ambient operating temperature: | -20 °C to 60 °C | -20 °C to 60 °C |
| Ambient storage temperature:   | +10°C to +30°C  | +10°C to +30°C  |
| Number of cells:               | 40              | 40              |
| Protection rating:             | IP54            | IP54            |
| Weight:                        | ≈ 2.4 kg        | ≈ 2.5 kg        |

We reserve the right to modify the design and technology of our products to incorporate the latest developments. Please retain this operating manual for future reference. This operating manual is also available to download from our website [www.ansmann.de](http://www.ansmann.de)

### The key elements at a glance:

#### Battery

|                   |   |
|-------------------|---|
| Battery (housing) | 1 |
| Key               | 2 |
| Charger socket    | 3 |
| On/off button     | 4 |
| LCD display       | 5 |

#### Motor

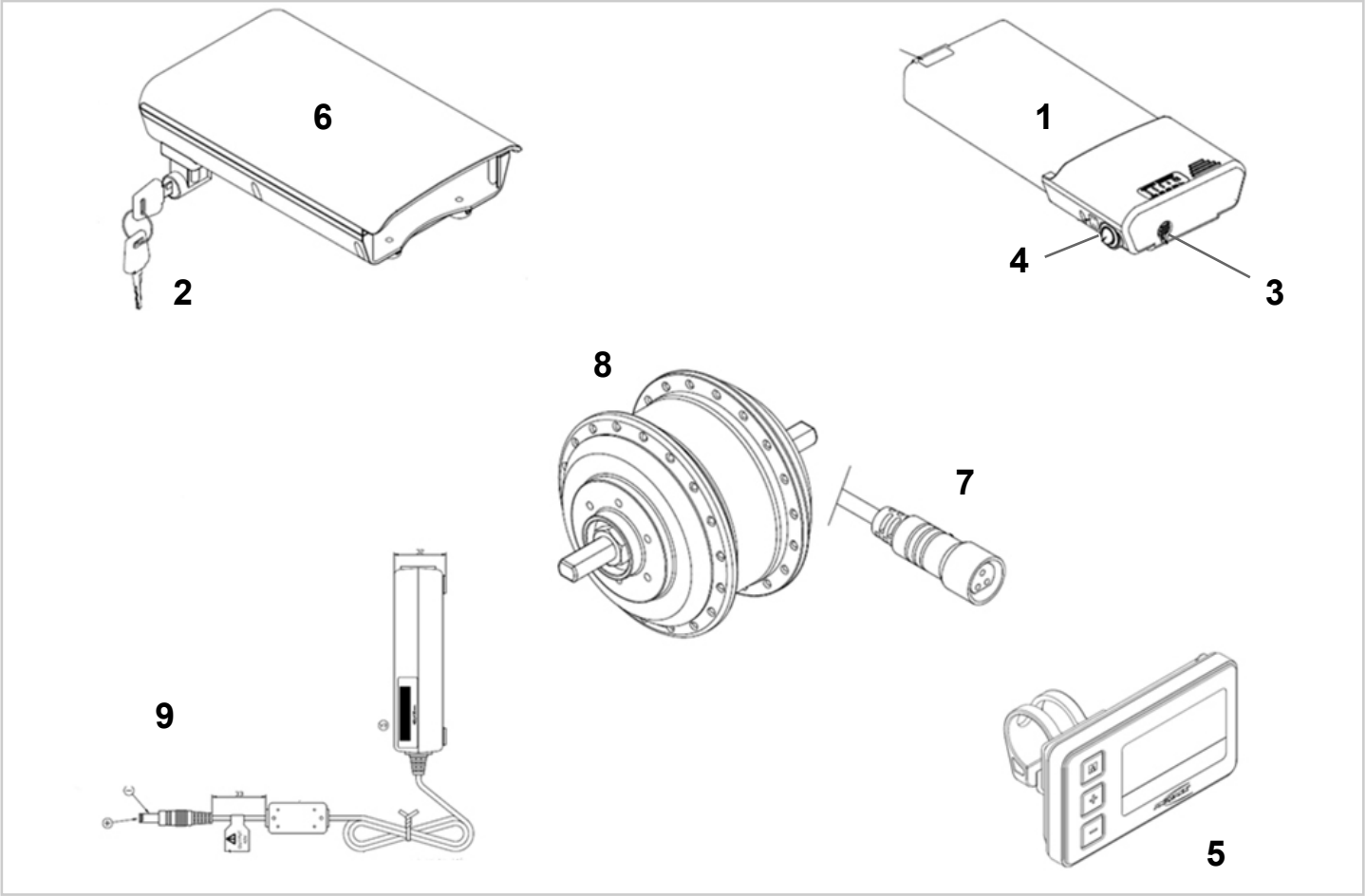
|                       |   |
|-----------------------|---|
| Motor cable connector | 7 |
| Motor                 | 8 |

#### Charging device

|                          |   |
|--------------------------|---|
| Charging cable connector | 9 |
|--------------------------|---|

#### On the add-on bike

|              |   |
|--------------|---|
| Battery slot | 6 |
|--------------|---|

### 3.0 Safety Advice/Precautions

#### **WARNING!**

- Read and observe the following safety instructions and precautions before activating the battery and before starting the charging process.

#### **DANGER!**

- Failure to comply with the safety precautions and instructions may damage the product or result in electric shock, fire and/or serious injuries.
- The lithium-ion battery contains chemical substances, which may cause hazardous reactions if the safety instructions specified here are disregarded.

#### **CAUTION!**

- The manufacturer accepts no liability for damage resulting from non-compliance with these instructions.

#### **Safety instructions and precautions on the use of the battery:**

#### **WARNING!**

- Before using for the first time, the battery should be fully charged.
- The battery must only be used at temperatures between -20 °C and 60 °C.
- The battery must not be exposed to heat (e.g. radiators) or fire. External heat exposure can lead to the battery exploding.
- In the (unlikely) event of the battery overheating or catching on fire, you must ensure that the battery does not come into contact with water or other liquids. The only suitable extinguishing agent that is recommended by the cell manufacturers is sand.
- Your add-on bike uses power in all operating modes. Therefore, if possible, charge the battery after every use.
- The battery must only be used to supply energy to the Ansmann components. Any other use requires the written authorisation of the manufacturer.
- The battery must not be opened or taken apart. Improper opening or wilful destruction of the battery involves the danger of serious injury. In addition, opening the battery voids any warranty claim.
- Never connect the battery contacts in the socket [3] to metallic objects, or ensure that the contacts never come into contact with metallic objects (for example with metal filings).
- If the socket [3] is dirty, clean it with a clean and dry cloth.
- Never immerse the battery in water.
- The useful life of the battery depends, among other things, on its storage location. Consequently, never leave the battery (irrespective of whether the battery is installed in or has been removed from the add-on bike) in hot locations for prolonged periods. In particular, use the boot of a car parked in the sun only for transport rather than as a storage location.

- The battery must not be subjected to mechanical impact. If, for example, the add-on bike overturns and the battery thereby hits the ground, the battery must be checked by the manufacturer. Contact your specialist dealer in this regard. You must not continue to use a damaged battery.
- If damaged or defective, the battery must be singled out and checked. Please contact your specialist dealer and discuss with them the next steps with regard to return and repair. The defective/damaged battery must not be used again or opened.
- Always ensure that the battery is kept clean and dry.

#### **Safety instructions and precautions on how to store the battery:**

#### **WARNING!**

- Protect the battery immediately upon separating from the battery charger or motor. Never allow any moisture or foreign particles (e.g. metal fragments, small nails, filings or other conductive metals) to get into the battery.
- Do not expose the battery to moisture of any kind during storage (water, rainwater, snow, etc.).
- Before storing it, charge the battery and check its charge status every 3 months.
- Store the battery in a cool and dry location where it is safe from damage and unauthorised access.
- To achieve the optimum battery service life, store the battery at a temperature from 5°C to 25°C and at a humidity of 0% to 80 %.
- Do not keep your battery in places where the temperatures above 45°C or below than -20°C.
- Avoid direct sunlight.
- When the battery is not in use, recharge it at least every 12 weeks to prevent damage to the battery.
- Check the charge status of the battery every 3 months during storage and charge it to 70 per cent if necessary.

#### **WARNING!**

#### **Safety instructions and precautions on the charging process:**

- Only charge the battery in a ventilated, dry and dust-free environment.
- Do not charge the battery in the presence or vicinity of flammable liquids or gases.
- Do not expose the battery to moisture of any kind during charging (water, rainwater, snow, etc.).
- Do not carry out the charging process in rooms where moisture may affect the battery.
- The battery must only be charged at temperatures between 0 °C and 40 °C. If you attempt to carry out a charging process outside of this temperature range, the battery mechanism automatically switches off the charging process. The battery reaches its maximum service life if it is charged at temperatures between 10 °C and 30 °C.
- Only use the designated charger to charge the battery. Your specialist dealer can provide the relevant information.

**⚠ WARNING!**

- Using an unsuitable battery charger can lead to malfunction and result in the battery having a limited service life. There is also a danger of fire and explosion.
- When the charging process is complete, disconnect the charger from the mains socket first before disconnecting it from the battery.
- Ensure adequate air circulation as soon as the battery is charged. In principle, only charge the battery under supervision.
- Damaged batteries must not be recharged or used any further.
- Damaged battery chargers (damage to the plug, housing, cable) must not be used.

**Safety information and precautions on how to transport and ship the battery:****⚠ WARNING!**

- Lithium-ion cells are used in the Ansmann battery. Transport and shipping of the battery are, therefore, subject to all applicable statutory requirements, which must be strictly observed. For instance, a defective battery must never be transported by plane.
- If your battery is defective, take it personally to your specialist dealer as shipping lithium-ion batteries by post or other carriers is subject to strict rules and regulations. Once again, we recommend that you contact your specialist dealer.
- As transport regulations may change annually, we strongly recommend that you consult your travel operator, airline or shipping line before you set out on your trip in order to learn about the current applicable regulations. A defective battery must not be taken on a plane or put in your luggage.

**⚠ CAUTION!**

- If your battery is fitted to the add-on bike when being transported, relaxed transport regulations apply in accordance with UN3171.
- Make sure you keep the packaging container of the battery in case you need to transport it.
- Discuss transportation with your specialist dealer prior to shipping.

**Safety information and precautions on the charger:****⚠ WARNING!**

- Before starting the charging process, read and follow all instructions and precautions enclosed with the charger and the following precautions and safety information.
- Only use the designated charger to charge the battery. Your specialist dealer can provide the relevant information.
- Using an unsuitable battery charger can lead to malfunction and result in the battery having a limited service life. There is also a danger of fire and explosion.
- The charging process ends automatically as soon as the battery is fully charged. This avoids overcharging.
- When the charging process is complete, we recommend disconnecting the charger from the mains socket first before disconnecting it from the battery.

- Never use a charger that is not recommended by your specialist dealer.
- Do not expose the charger to moisture of any kind during charging (water, rain water, snow, etc.).
- Do not carry out the charging process in rooms where moisture may affect the charger.
- Be aware of condensation. If the charger is brought from a cold into a warm room, condensation may form. In this case, refrain from using the charger until all condensation has dissipated. Please note that this may take several hours.
- Never carry the charger by its power cable or the charger cable.
- Never tug at the power cable to disconnect the charger from the mains socket.
- Never subject cable and plug to any pressure. Overstretching or bending the cable, pinching a cable between a wall and a window frame or placing heavy objects on a cable or a plug may result in electric shock or fire.
- Lay the power cable and the attached charging cable so that nobody can step on it or trip over it and so both cables are protected against any other harmful effects or stress.
- Do not operate the charger if the power cable, the charging cable or the plugs attached to the cables are damaged. Damaged parts must be replaced immediately by the authorised specialist dealer.
- Do not use or disassemble the charger when it has received a hard blow or was dropped or damaged in another way. Take the damaged charger to a specialist dealer who has been authorised to perform repairs.
- The charger must not be used by children.
- Never attempt to disassemble or modify the charger.
- Do not cover the charger during the charging process or place any objects on top of the device.
- Never connect the terminals of the charging plug with any metal objects.
- Ensure that the plug is firmly inserted in the socket.
- Never touch plugs with wet hands.
- Do not use the plug of the charger and/or the mains plug if they are wet or dirty. Before inserting it, clean the plug using a dry cloth.

## 4.0 Battery

### Start Up:

#### Information on the operating modes:

Essentially, the battery has two operating modes. It is either in "Active mode" or in "Off mode". In active mode the battery consumes at least 50 mA per hour (consumption of the battery's own electronics and display).

#### Inserting the battery:

(Fig.4.1).

- Place the battery [1] into the battery slot [6] mounted on the add-on bike.
- Lock the battery [1] by carefully turning the key [2] clockwise until it stops.
- The battery can no longer be removed from the battery slot.
- Remove the key [2] from the battery [1].

#### Switching on the battery:

The battery is switched on by operating the on/off rocker switch, (Fig.4.2).

Your add-on bike is now ready for use.

If the battery cannot be switched on, the cell voltage may be too low. In this case connect the charger, (see next page) and then press the on/off button [4]. The battery will be charged for a minute.

#### Battery capacity indicator:

You can check the battery capacity at any time using the LED display. Press the button, (A), briefly to display the charge status, (Fig.4.3).

#### Detaching the battery:

#### Switching off the battery:

The battery is switched on by operating the on/off rocker switch, (Fig.4.2).

#### Removing the battery:

Reverse the insertion procedure, (Fig.4.1).

Insert the key [2] in the lock on the battery [1].

Carefully turn the key [2] anticlockwise until it stops. The lock is now released.

Pull the battery [1] downwards out of the slot and then remove it completely.

Store the battery in a safe clean place.

Fig.4.1


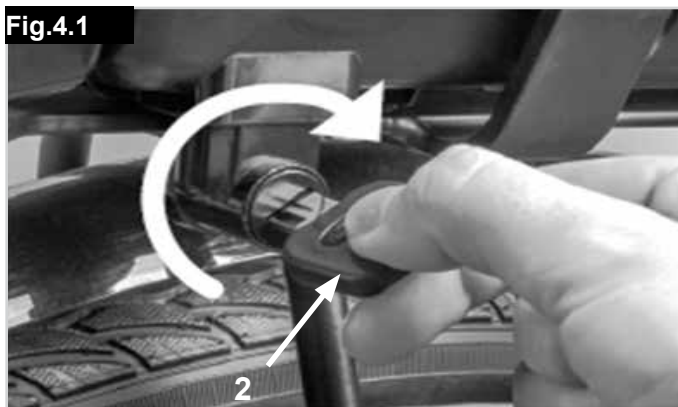
Fig.4.2

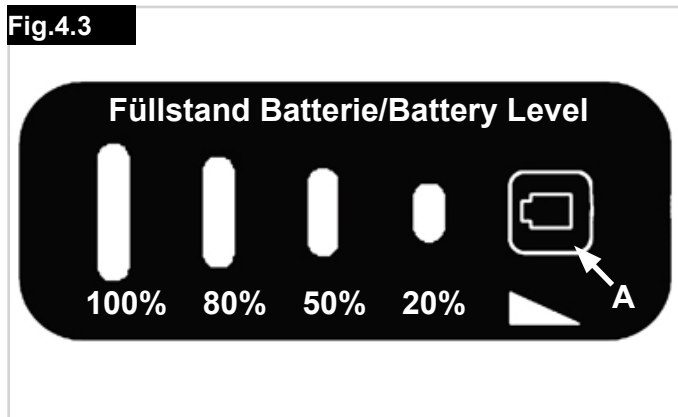Fig.4.3

### Charging the battery:

Fully charge the battery before its first use. The battery capacity upon delivery is generally 30%. The battery can be charged at any capacity without adversely impacting its service life. The battery achieves its maximum service life when it is charged at an ambient temperature of between +10 °C and +30 °C

### Connecting the battery charger:

The battery [1] does not need to be removed from the add-on bike for charging, it can stay where it is. Then, proceed as before.

Insert the plug [9] of the battery charger into the socket [3] on the battery, (Fig.4.4).

Carry out the charging process according to the specifications in the charger operating manual.

In addition, follow the instructions on the charging process given in section 3.0.

Before inserting the plug [9] in the socket [3], ensure that both parts are clean and there are no metallic particles on them. If you detect such particles, use a dry and clean cloth to remove them.

### Charging process:

When charging the battery follow the instructions in the charger operating manual.

Also observe the safety information and precautions provided in section 3.0

### LED displays during the charging process:

During charging, the LED display on the battery shows the charging level.

Press the button, (A), briefly to display the charge status, (Fig.4.5).

### CAUTION!

- Do not leave the charger connected to the mains socket any longer than necessary.
- When the charging process is complete, disconnect the charger from the mains socket first before disconnecting it from the battery.
- Before using the device, always check the charge status of the battery.
- The battery should be fully charged prior to use in order to provide motorised support at all times.
- The battery may only be charged in a dry room at temperatures between 0° and 40° C.
- We recommend charging the battery fully before long periods of storage, eg, before a winter break

Please observe the instructions in the operating manual enclosed with the charger.

Observe the safety information and precautions regarding the battery provided in sections 2.1 to 2.5 of this operating manual.

Fig.4.4


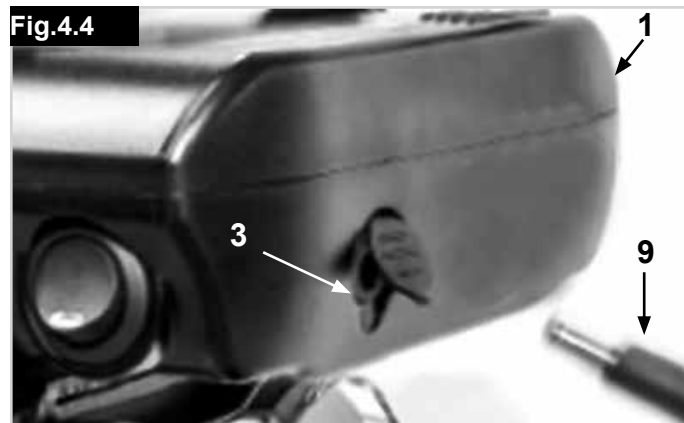
Fig.4.5
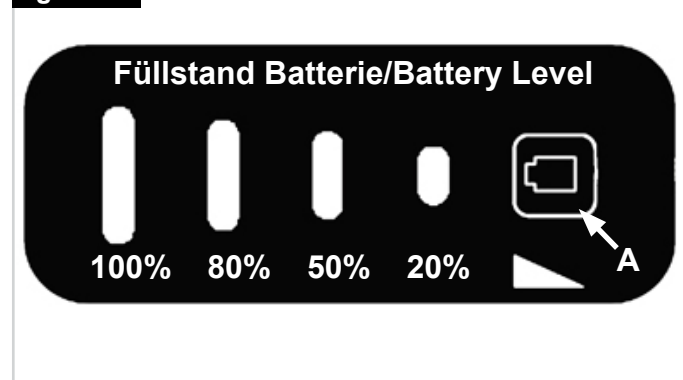
### Fuse Replacement At The Battery:

(Fig.4.5 4.6).

If your e-bike provides no motor support and the display is blank, check the fuse at the battery.

Proceed as follows:

- Open fuse cover, (A).
- Pull out and check the fuse, (B), replace it, if necessary
- Insert the fuse again
- Close fuse cover

#### CAUTION!

Only use the same fuse model with the same electrical values (15A/80V).

### Cleaning the battery:

#### CAUTION!

When cleaning the device, never use cleaning benzine, thinner, acetone or similar agents.

Instead, use only conventional household cleaning agents and disinfectants (eg. isopropyl alcohol).

The plug of the charging cable on the battery charger [9] and the charger socket [3] on the battery may only be cleaned using a dry cloth.

The battery must never be sprayed with a steam cleaner or similar device.

### Battery Storage:

We recommend storing the unused battery in a dry place at temperatures between 10°C and 30°C.

- Do not keep your battery in places where the temperature exceeds 45°C or is less than -20°C.
- Avoid direct sunlight.
- Avoid storage in humid areas to prevent corrosion on the plug contacts.
- When the battery is not in use, recharge it at least every 12 weeks to prevent damage to the battery."

Fig.4.5


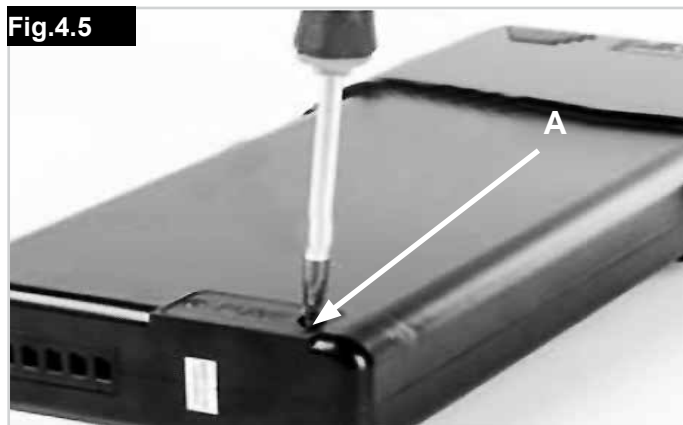
Fig.4.6
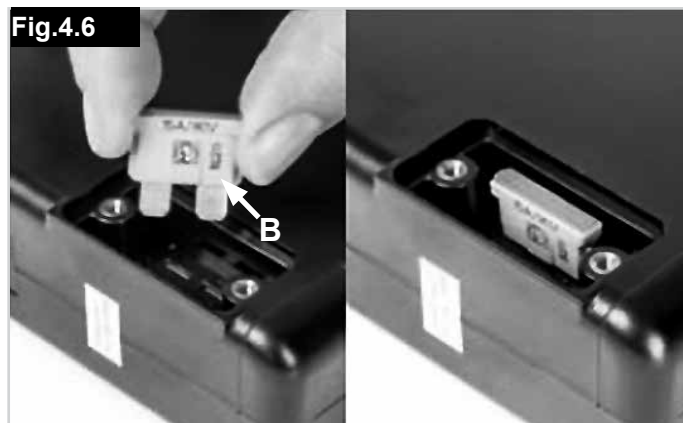
## 5.0 Disposal

### Disposal:

Electrical and electronic equipment need to be disposed of separately to general household waste at specific state-provided locations. The correct disposal and separate collection of used appliances serves to prevent potential damage to health and the environment. It is a requirement for the re-utilisation and recycling of used electrical and electronic equipment.

Detailed information on the disposal of your used equipment can be obtained from your local authority, your waste disposal service, the specialist dealer from which you purchased the product, or your sales contact.

This information only applies to equipment which is installed and sold in the countries of the European Union and which is subject to the European directive 2002/96/EC. In countries outside of the European Union, deviating conditions apply to the disposal of electrical and electronic waste.

Please also note the information on the rating plate on the bottom of the battery:

Protect the battery from temperatures above 60°C

Keep the battery away from open flames

CE-Sign (Conformity Europe), this rechargeable battery is built in conformity with European standards

China RoHS marking: There are dangerous materials used. The number in the logo indicates how many years the product can be used as intended.

Cells contain lithium

Do not dispose of the battery pack with household waste.

Sign and polarity of the charging socket/the charger

6.0 Battery Charger

|                                                                                   |                                    |
|-----------------------------------------------------------------------------------|------------------------------------|
| Model                                                                             | C060L1001/ACK4201                  |
| Application                                                                       | for 36V LI-Ion Battery             |
| Input                                                                             | 100V-240V~50/60Hz 1.2A             |
| Output Voltage                                                                    | 42.0V +/- 0.5V                     |
| Output Current                                                                    | 1.35A +/- 0.2A                     |
| Power Output                                                                      | max. 60W                           |
| Colour                                                                            | black                              |
| Dimensions                                                                        | 115 x 59 x 32 mm                   |
| Weight                                                                            | 280g                               |
| Power Cable                                                                       | 1.8 m / 3 different countries plug |
|  |                                    |

Your charger is delivered with the typical local mains plug fitted. If you need replacement or alternative plug designs, please contact Sunrise Medical or your local authorised dealer.

**NOTE:** When using the battery charger, please note the printed instructions on the battery charger and the following provided information.

 **WARNING!**

- This charger must be connected only on sockets with protection class I.
- Please read the following instructions carefully and give these instructions, if necessary, to third parties.
- Use only the original provided ANSMANN-Charger for your battery pack.
- Installing other than specifically designated battery packs may cause problems / injuries, because the power and Voltage protection are only designed for ANSMANN batteries.
- Do not attempt to open the charger under any circumstances.
- If necessary, clean the charger with a dry cloth.
- The charger must only be used in dry areas. Protect it from rain and moisture, fire and high temperatures.
- Take care to avoid temperature changes that may induce condensation.
- Remove the charger from the mains on completion of charge.
- During the charging process do not cover the charger.
- This appliance is not intended for use by people with reduced physical, sensory or mental capabilities or lack of experience and / or knowledge (including children), unless they are supervised by a responsible person for their safety or receive from this person instruction on how the appliance is to be used.
- Children should be supervised to make sure they do not play with the appliance.
- If the power connection of this device is damaged, it must be replaced by a special connecting cable, which is available from the manufacturer or service agent.

7.0 Liability

**Liability:**

- The manufacturer is not liable in any instance if:
- the battery has been / is incorrectly handled.
  - the battery has been / is commissioned in contravention to the instructions in this operating manual.
  - the battery has been / is operated with insufficient battery charge.
  - the battery has been / is repaired or otherwise modified by a person not authorised to carry out such work.
  - the battery has been / is used contrary to the intended use.

Sunrise Medical GmbH & Co. KG  
Kahlbachring 2-4  
69254 Malsch/Heidelberg  
Deutschland  
Tel.: +49 (0) 7253/980-0  
Fax: +49 (0) 7253/980-222  
kundenservice@sunrisemedical.de  
www.SunriseMedical.de

Manufacturer of the motor battery and display:  
ANSMANN AG  
Industriestrasse 10  
97959 Assamstadt  
Tel.: +49 (0) 6294 4204 0  
Fax: +49 (0) 6294 4204 4400  
info@ansmann.de  
www .ansmann.de

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