EXCEL GALAXY II

How To Adjust the Clock Time, Temperature and Speed Options



- 1) Turn off the key
- 2) Press and hold both head lights and half speed buttons
- 3) Turn on the key switch
- 4) When AM/PM flash, release both buttons
- 5) To change setting of each function use the indicator light switches
- 6) Press the Hazzard lights switch to confirm each function as below;

Clock - AM or PM

Clock – Hour

Clock - Minute

Temp – Fahrenheit or Celsius

Speed – Kilometers or Miles

Please Note: If nothing is pressed after 10 seconds it will resume back to the normal display screen.

